


TruCric

User Manual

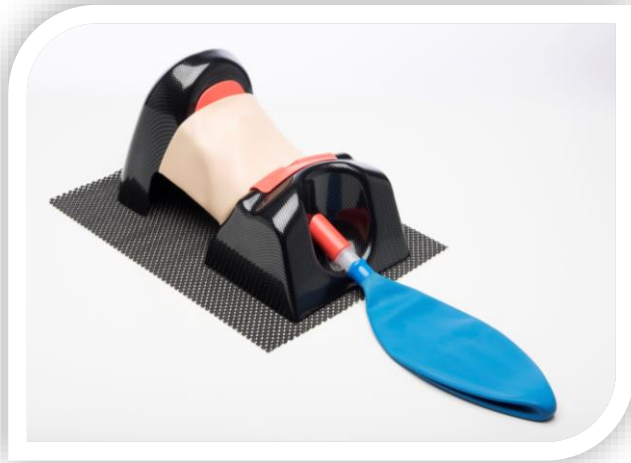

Product Code: TCRIC1

Size: 40 x 24 x 22 cm

Ship Weight: 2 kg

Package contents:

- 1 Black carrier case
- 1 USB pen drive user manual
- 1 Bottle of lubrication
- 1 Adult Lung Bag Set
- 10 Larynx inserts (1 supplied on the manikin and 9 spare)
- 2 wrap around neck skins (1 supplied on manikin and 1 spare)
- 1 non-slip mat

Before you begin....

- Secure the model to a suitable surface by adjusting the screw on the suction cups (the surface needs to be non-porous to ensure sufficient suction)
- If used on a surface not suitable for the suction cups, then please use the non slip mat provided to stop the product from sliding during use
- Check the lung bag is attached to the airway tube
- Check to ensure that the Larynx is in correct position with the skin secured in place
- Spray generous amounts of lubrication on the equipment at all times to ensure product efficiency

- You are now ready to use the manikin

The TruCric is ideal for training in surgical airway techniques including:

- Needle and surgical cricothyroidotomy
- Percutaneous tracheostomy
- Air Jet ventilation

Consumable Guide:

Replacing the larynx insert:

Remove the neck skin and larynx insert. When inserting the new larynx please use the notches in the manikin to fix in place.



When replacing the neck skin ensure that it is passed through the pre-defined gaps on the base and that the skin rests on the Larynx insert to give a prominent definition

Care and Maintenance

Store in clean, dry conditions away from heat and direct sunlight; avoid contact with metals, solvents, oils or greases and strong detergents. When the product is not in use please store in the black carrier case provided.

Thoroughly wash the neck area in warm water. Please use warm soapy water or something similar until all visible foreign matter and residue is removed.

Mild detergents or enzymatic cleaning agents may be used on the airway in accordance with the manufacturer's instructions and at the proper dilution. The detergent must not contain skin or mucous membrane irritants.

Please do not use any of the following when cleaning the AirSim product range

- Germicides, disinfectants, or chemical agents such as glutaraldehyde (e.g. Cidex®),
- Ethylene oxide, phenol-based cleaners or iodine-containing cleaners

Warranty

TruCorp warrants this unit to be free of defects in materials and workmanship and to give satisfactory service for a period of 1 year from the date of delivery.

This ensures that our customers receive maximum coverage on each product. If the unit should malfunction it must be returned to the factory for evaluation. Upon examination by TruCorp, if the unit is found to be defective it will be repaired or replaced at no charge. However this warranty is VOID, if; the unit shows evidence of having been tampered with or shows evidence of having been damaged by excessive heat, the use of sharp instruments, misapplication, misuse or other operating conditions outside of TruCorp's control. Components which wear or are damaged by misuse are not warranted and will be charged for if repair has been approved.

Please direct all warranty and repair requests/inquiries to:



The Mount Business Park
2 Woodstock Link,
Belfast, BT6 8DD

info@trucorp.com / rcolhoun@trucorp.com



Tel: +44 (0) 28 9073 7281 Fax: +44 (0) 28 9073 7282