

60680

MICROVASCULAR ANASTOMOSIS TRAINER



60681

MICROVASCULAR ANASTOMOSIS VESSELS (PACK OF 5)



60682

MICROVASCULAR ANASTOMOSIS JIG

CONNECTORS (x2)



EXTENDERS (x2)





FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

HOW TO SET IT UP:


Extenders and connectors are packaged separately, with jig base and lid

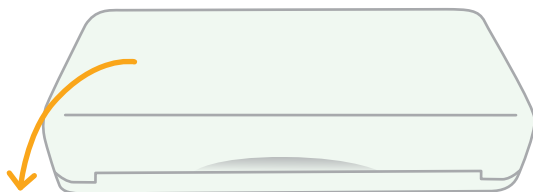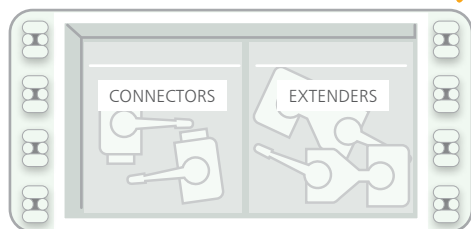


1 Connectors are inserted into the ridges on either side of the base at any position



2 Extenders can be inserted onto the connectors to lengthen the arms

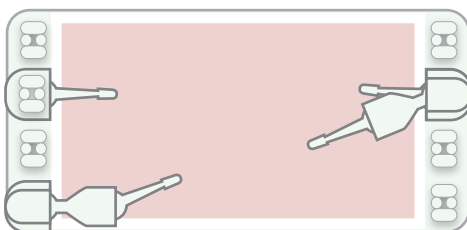

3 Insert a micro vessel onto the holding arms at any length of height required for the training session

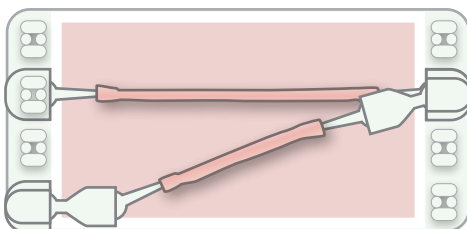

4 Perform varying anastomosis procedures