





USER GUIDE

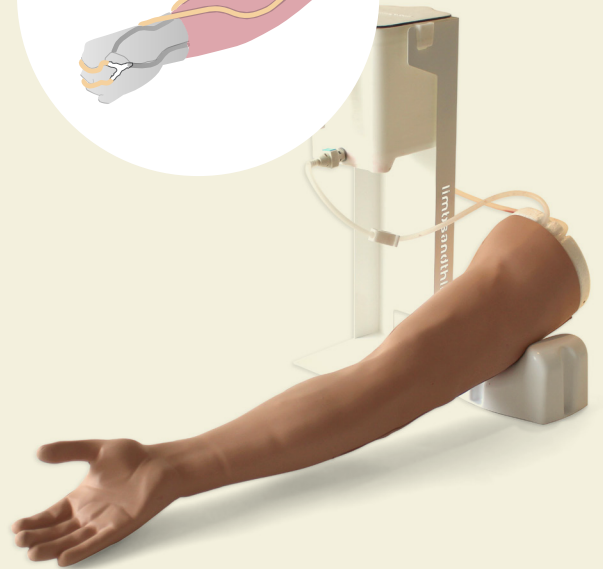

BASIC VENIPUNCTURE ARM

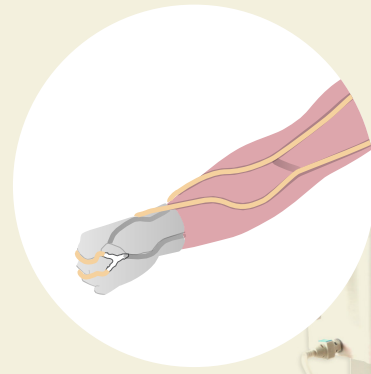

This product is available
in light and dark skin tones

70304

70305

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• ADDING BLOOD TO THE GRAVITY SYSTEM	5
• CONNECTING ARM TO GRAVITY SYSTEM	6
• EMPTYING GRAVITY SYSTEM & VEINS	7
• REMOVING/RE-ATTACHING SKIN	8
• CHANGING THE VEINS	10
CARE & MAINTENANCE	11
• CLEANING THE VEINS	11
• BEFORE STORAGE	11

TAKE A LOOK
AT OUR **VIDEO**




VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE
SUPPLIED

Venipuncture Arm

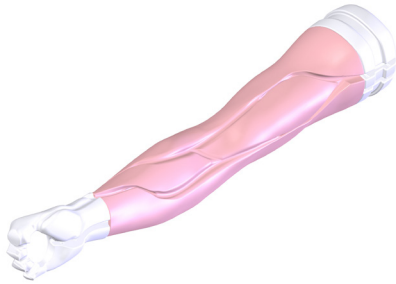

Venipuncture Arm Skin

70312 70311



Gravity System
Blood Supply

70316



Basic Venipuncture Arm Vein

70309




Lube Sachet (x3)




Venipuncture Arm Rest

00302

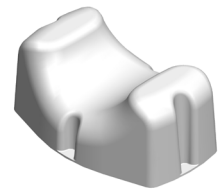

Concentrated Venous Blood

00020

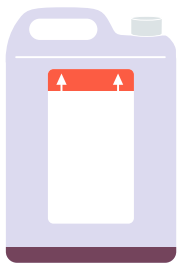

Venipuncture Arm
Infusion Tube

00335



Micropore tape



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR VENIPUNCTURE ARM

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Only use simulated blood supplied with this product. Use of non-recommended fluid products may weaken or damage the simulator.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Only use water-based lubricant.



Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not set up the product near electrical appliances or items damaged by water.



Do not wear any jewellery items.

Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.



Do not leave the product filled with blood when putting into storage.

STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.





VEINS CONTAIN LATEX


SET UP

ADDING BLOOD TO THE GRAVITY SYSTEM

Place the jug onto the stand

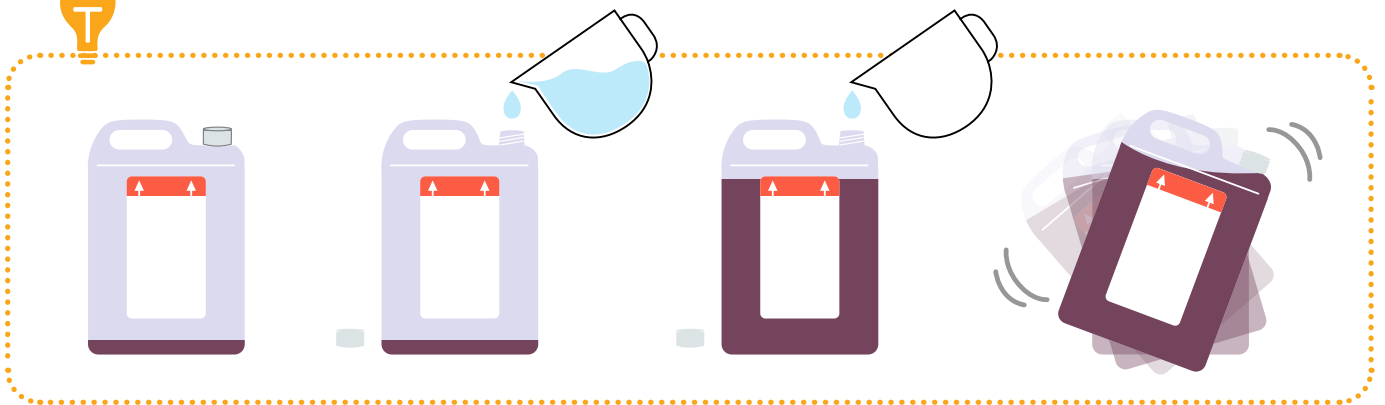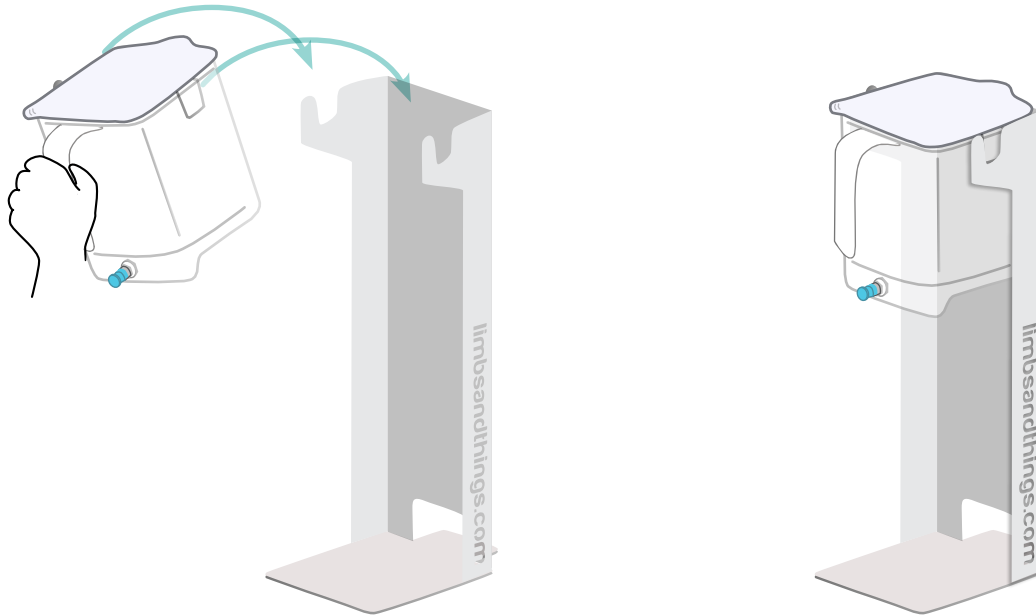

Fill the jug



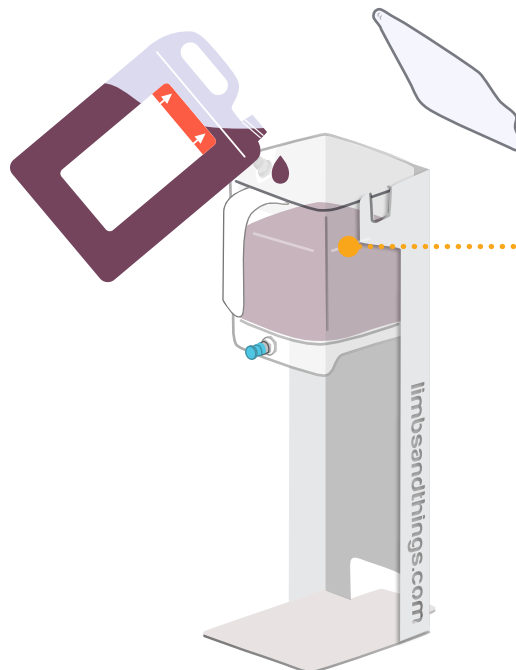


Between 1-1.5 liters

SET UP

CONNECTING ARM TO GRAVITY SYSTEM

Connect the tubes to the Gravity System

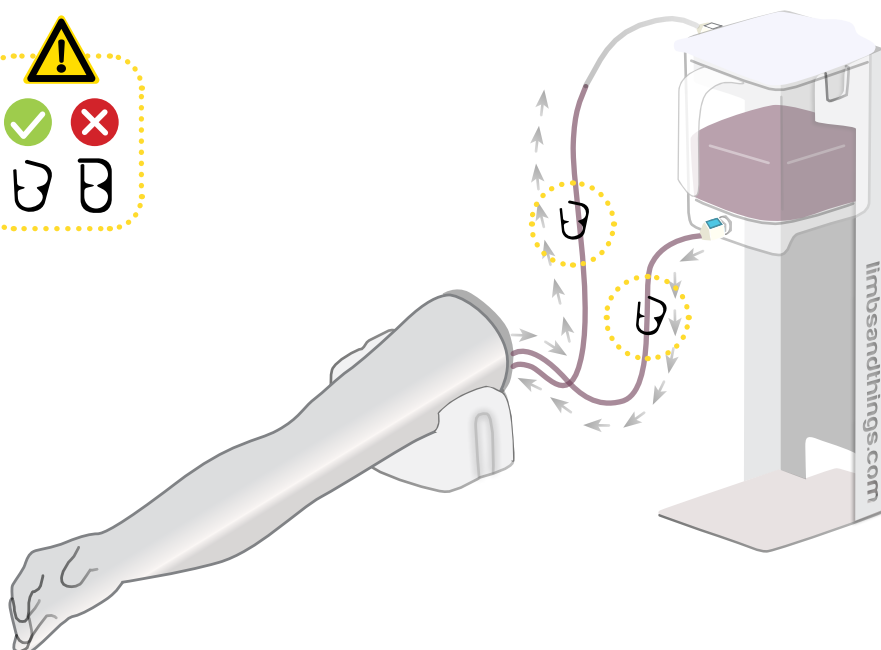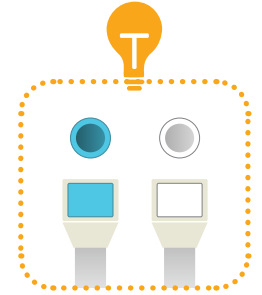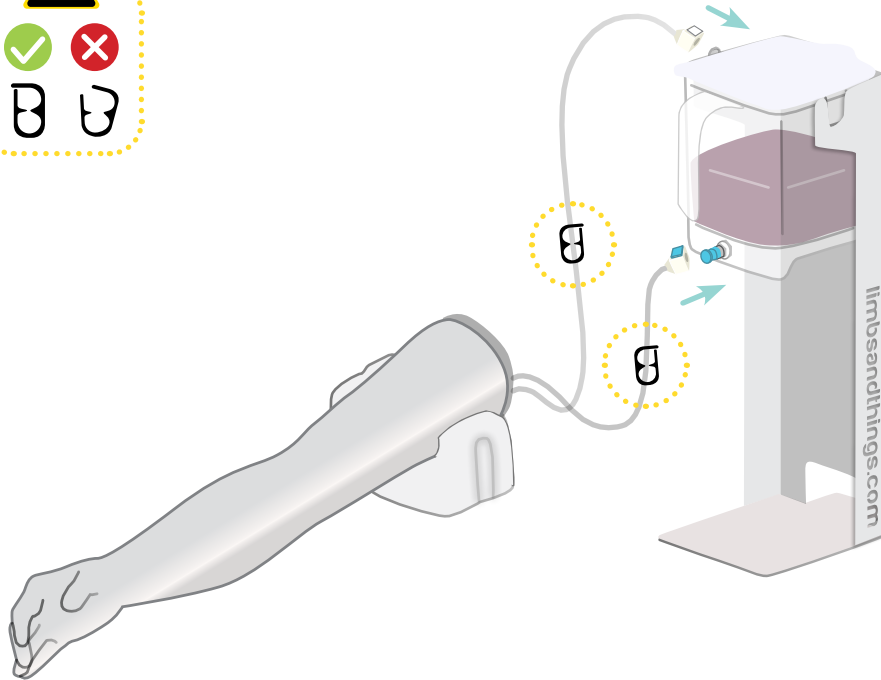


The Arm is ready to use

SET UP


EMPTYING GRAVITY SYSTEM & VEINS

Disconnect blue tube
and connect it to infusion tube

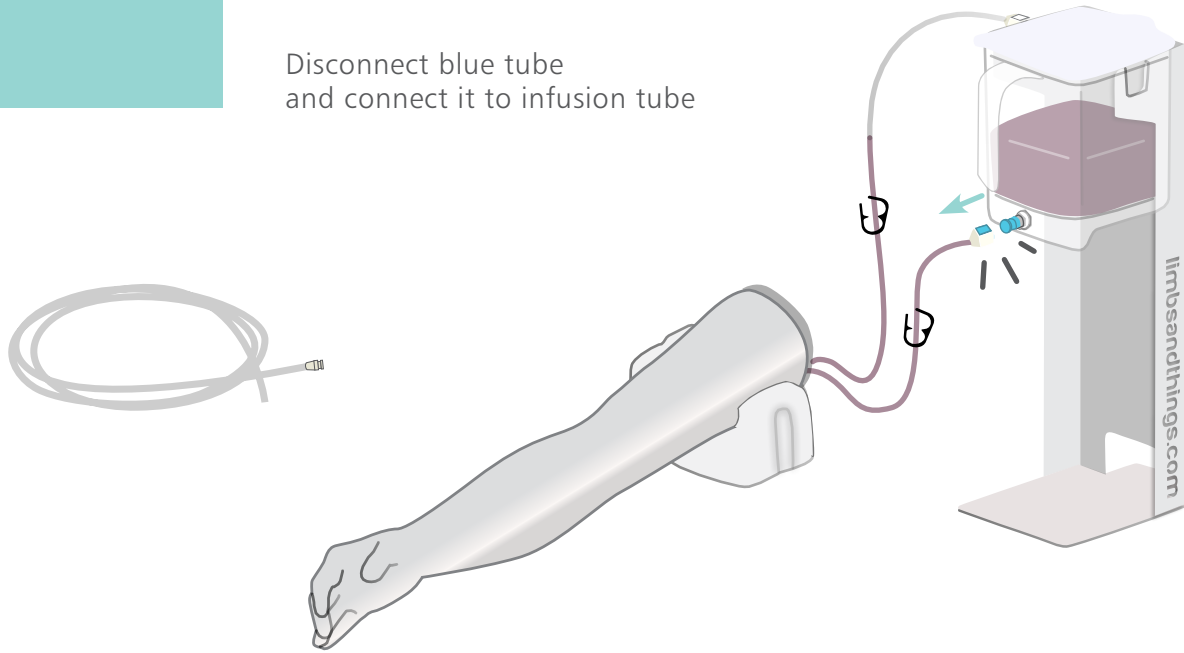

Remove all blood from the clear tubing.


Any remaining blood will cause model priming to be slower on next use.

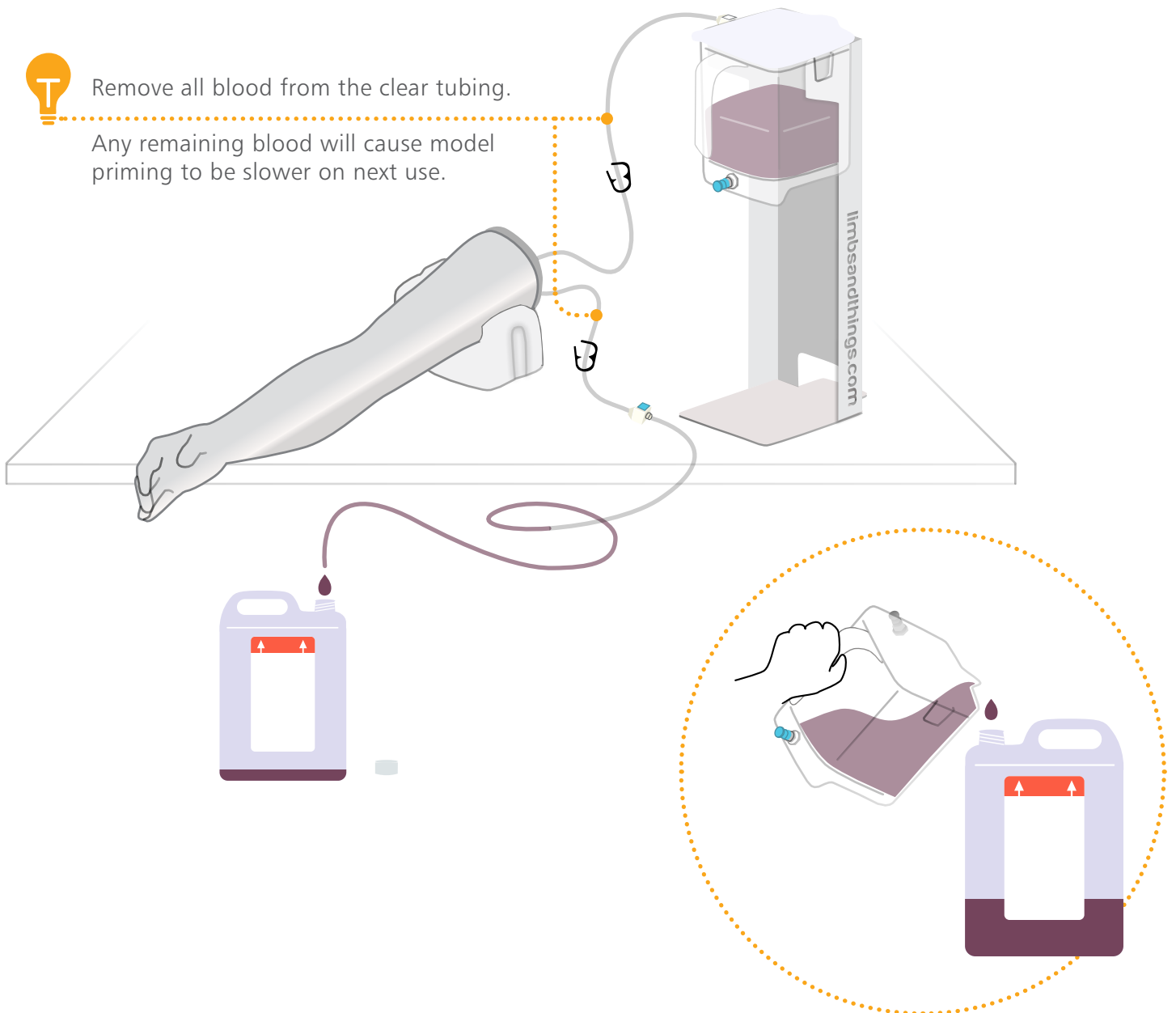

SET UP

REMOVING/RE-ATTACHING THE SKIN


Remove

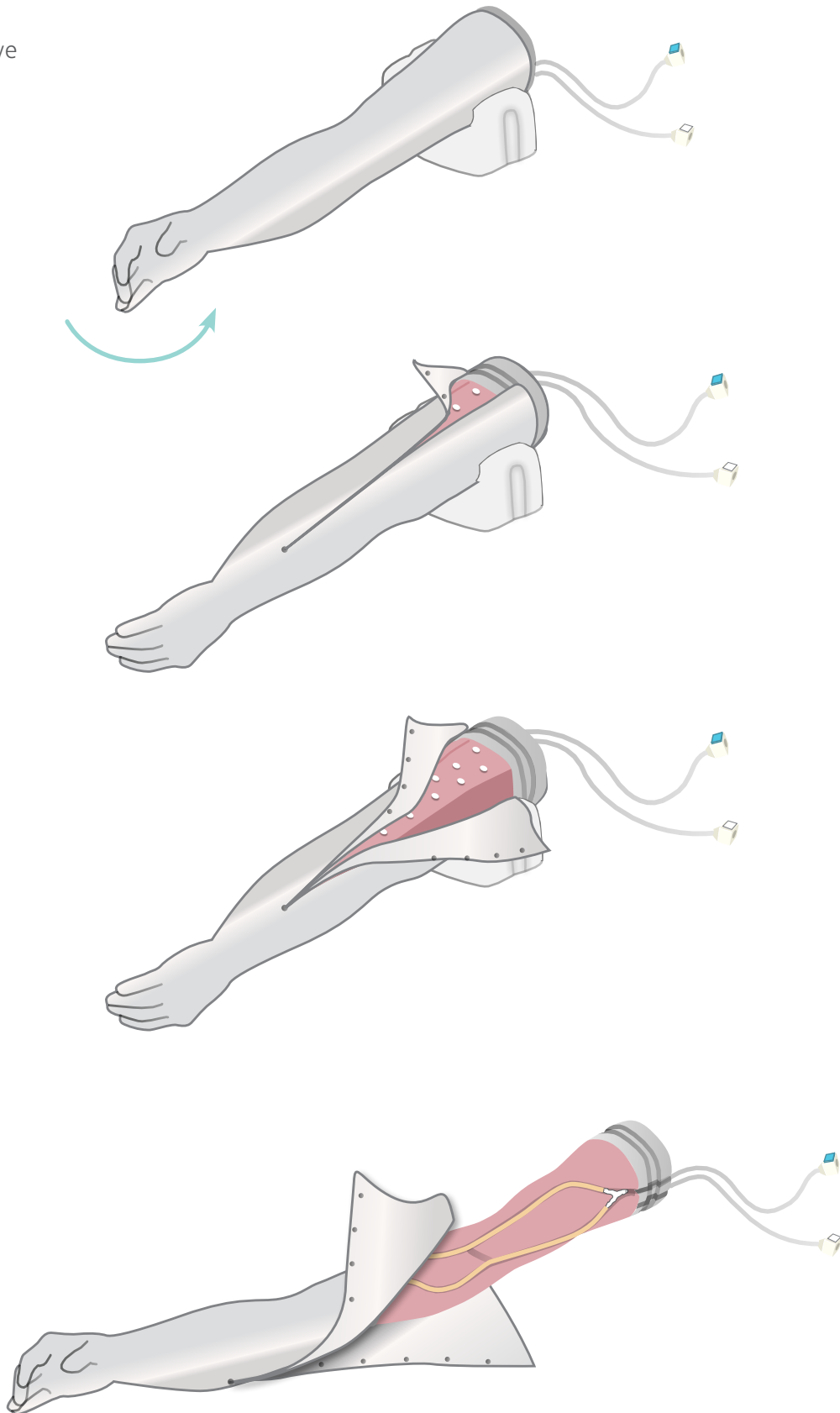

SET UP

REMOVING/RE-ATTACHING THE SKIN




Re-attach

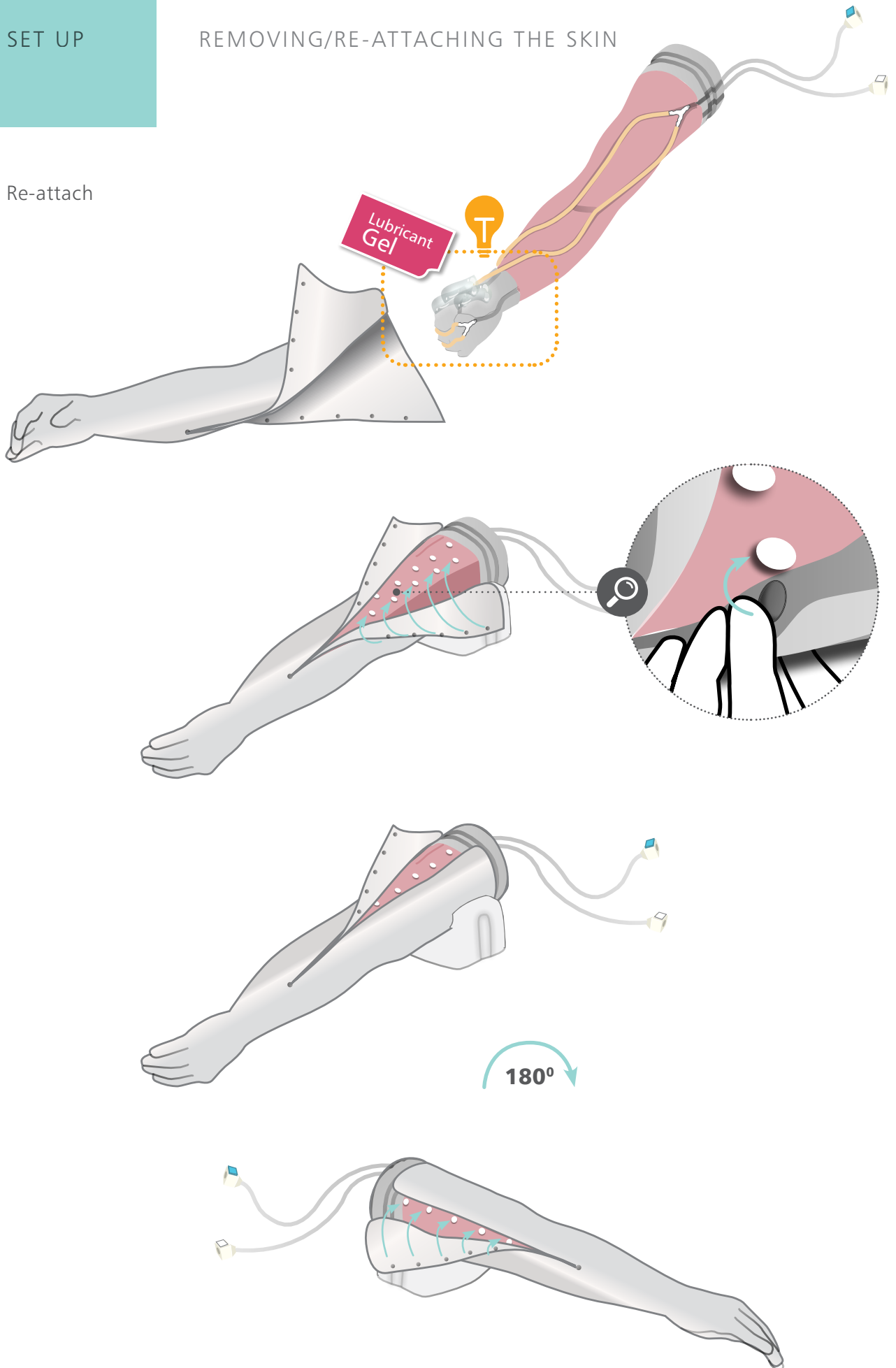


SET UP

CHANGING THE VEINS

Disconnect

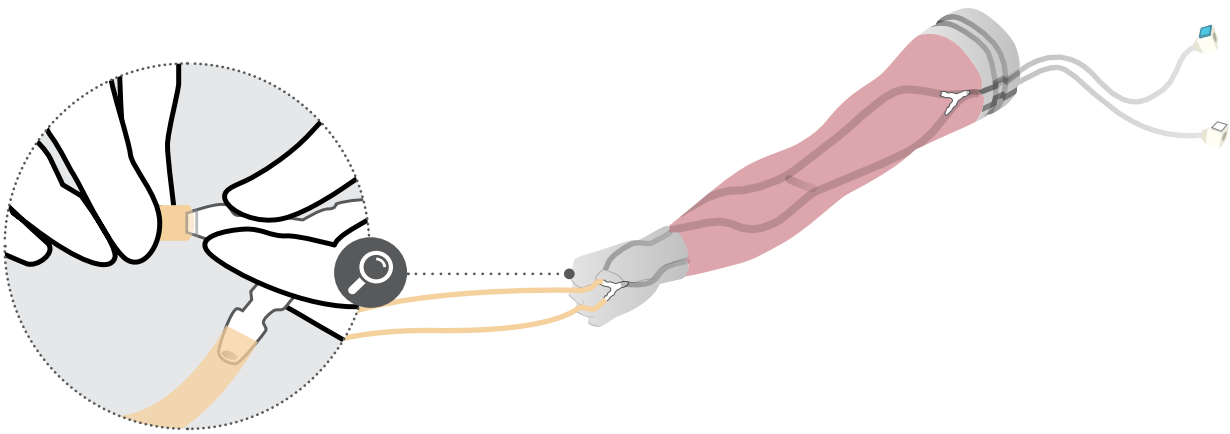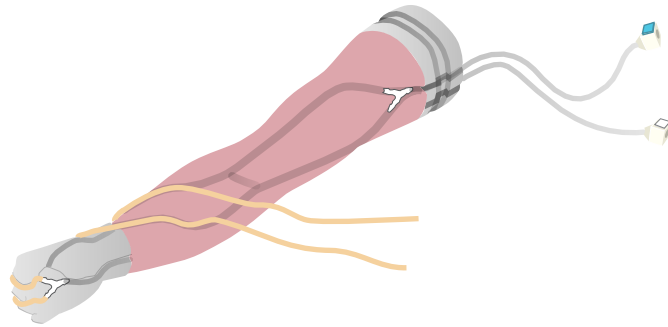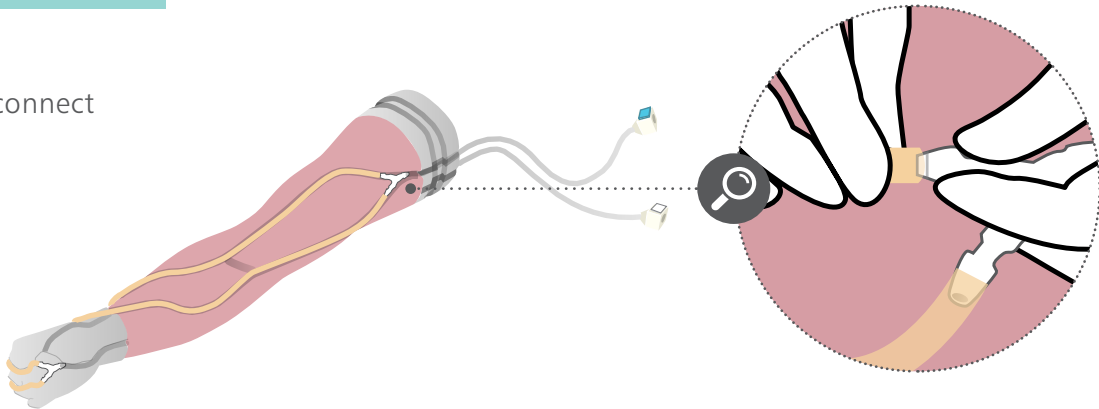


Attach new veins

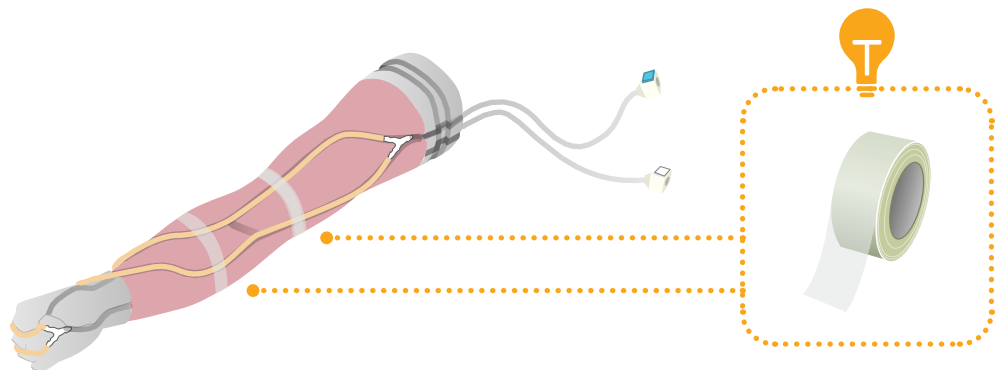

CARE & MAINTENANCE

CLEANING THE VEINS



For a more thorough clean

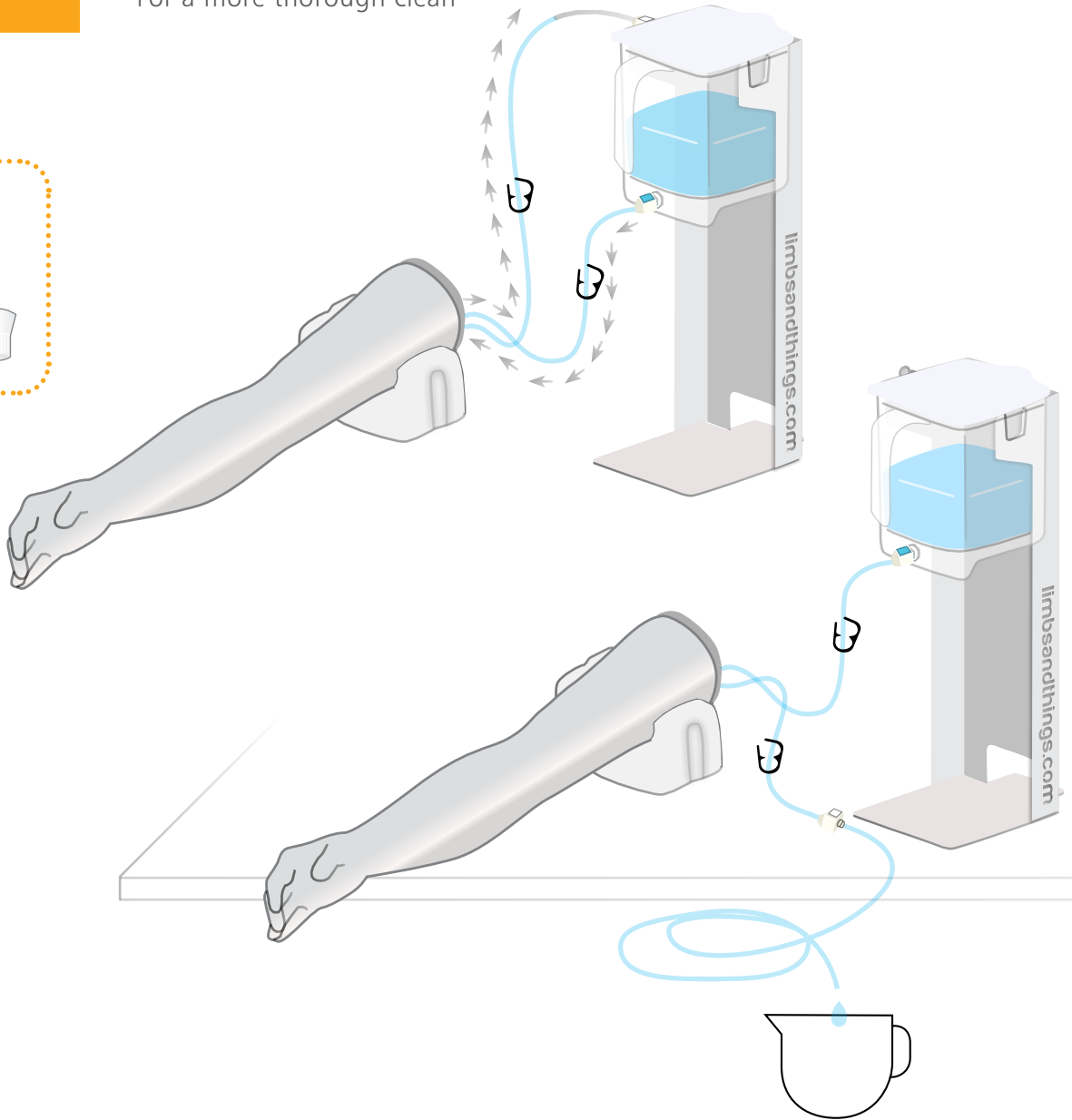

BEFORE STORAGE

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent

Do not leave the product filled with blood

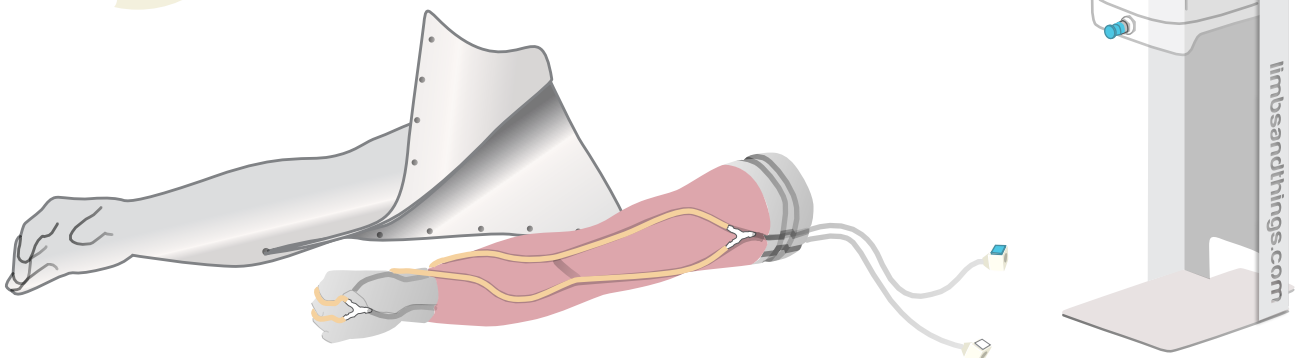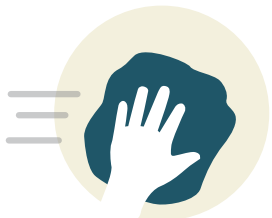

YOU MAY
ALSO BE
INTERESTED IN...

OPTIONAL TO PURCHASE:


ACF Vein Module (x2)

70306




Metacarpal Vein Module (x2)

70307

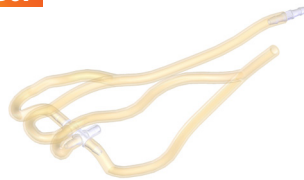

Basic Venipuncture
Arm Vein

70309

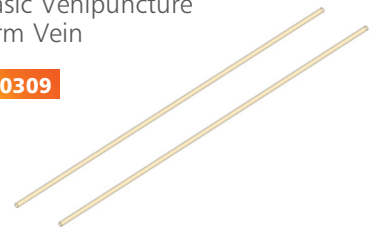

Basic Venipuncture
Arm Vein - Latex Free

70310



Limbs & Things
Lubricant

60869



Carry Case

00301

