


USER GUIDE





CLINICAL FEMALE
PELVIC TRAINER
(CFPT) MK 3

ADVANCED

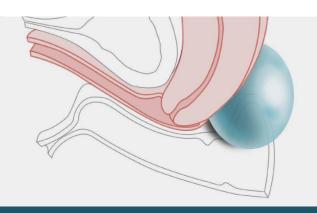

This product is available in light and dark skin tones

60905

60935

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
SETTING UP THE CLINICAL FEMALE PELVIC TRAINER	5
CHANGING THE UTERINE MODULES	6
TRAINING	7
BI-MANUAL EXAMINATION	7
CLEANING & STORAGE	7
CLEANING & STORAGE AFTER USE	7

TAKE A LOOK AT OUR **VIDEO**



VISIT THE WEBSITE limbsandthings.com

OR YOUTUBE CHANNEL youtube.com/limbsandthings

PACKAGE SUPPLIED

CFPT ABDOMINAL INSERT

60936



LOWER TORSO



BASE



CFPT MODULE 1 Normal - Nulliparous Cervix



TUBE OF AQUEOUS GEL

00293



CARRY CASE



CFPT MODULE 2

Large Fibroid - Nulliparous Ectropion Cervix



CFPT MODULE 3

Small Fibroid - Nulliparous Polyp Cervix



CFPT MODULE 4

Ovarian Cyst - Multiparous Cervix



CFPT MODULE 5

Retroverted - Multiparous Cervix



CFPT MODULE 6

10-12 Weeks Pregnant



CFPT MODULE 7

14-16 Weeks Pregnant



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR CLINICAL FEMALE PELVIC TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Only use water-based lubricant, such as the one supplied with this model.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Lubricate your examining fingers before insertion.



Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.





Do not wear any jewellery items.

Do not use if the skin is torn.





Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.






CHANGING THE UTERINE MODULES

1




LIFT THE ABDOMINAL INSERT UP FROM THE TOP OF THE TRAINER

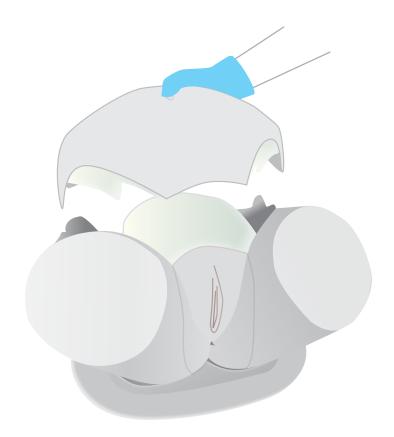

2



LIFT THE MODULE OUT USING BOTH HANDS

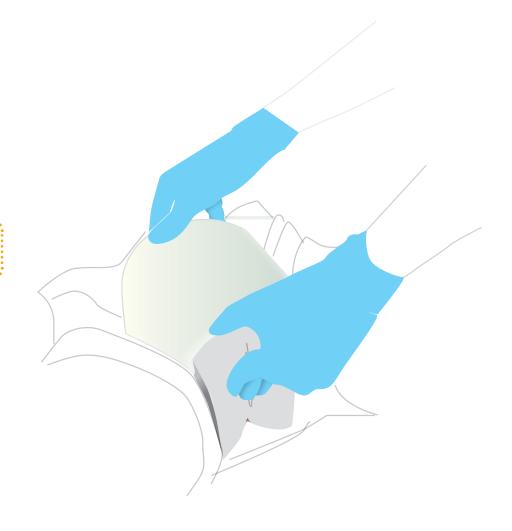

TRAINING


PERFORMING THE BI-MANUAL EXAMINATION

- Wear gloves at all times and remove all jewellery items.
- Lubricate prior to examination.







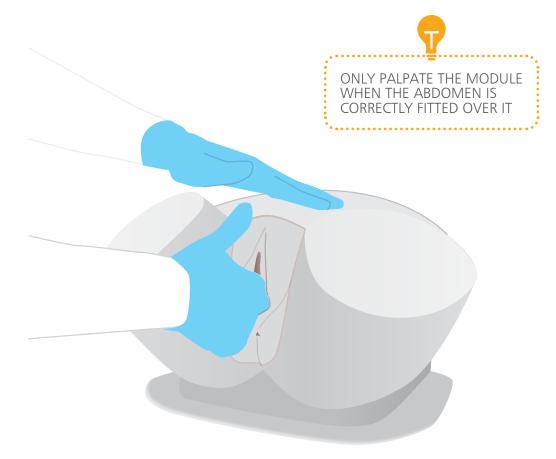

CLEANING & STORAGE

- Wipe off excess lubricant with a soft cloth
- Clean off any surface dirt with a damp, soft cloth, using only warm water and mild detergent



YOU MAY ALSO BE INTERESTED IN...



BREAST EXAMINATION TRAINER - STANDARD & ADVANCED

STANDARD

40200

41200



40201

41201







Part No: 065-963 Issue 5, May 2022 © 2020 Limbs & Things