



USER GUIDE



BREAST EXAMINATION TRAINER

BASIC

This product is available
in light and dark skin tones

40202

41202

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	2
SAFETY & CLEANING INSTRUCTIONS	3
SET UP	4
• SETTING UP PATHOLOGIES	4
• REMOVAL & REFITTING BREAST INSERTS	6
TRAINING	7
• FITTING THE BREAST SKIN FOR WEARABLE USE	7
CLEANING & STORAGE	7
• CLEANING & STORAGE AFTER USE	7

PACKAGE SUPPLIED

Wearable Examination Breasts



Breast Examination Pathologies

40204



Breast Examination Inserts

40205



Breast Pathologies Supports

40206



Breast Back Plates

40207



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR BREAST EXAMINATION TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

To stop the insert from sticking to the inside of the skin pour a small amount of talcum powder into the recess of the breast skin.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.




SET UP

SETTING UP PATHOLOGIES

1




180°

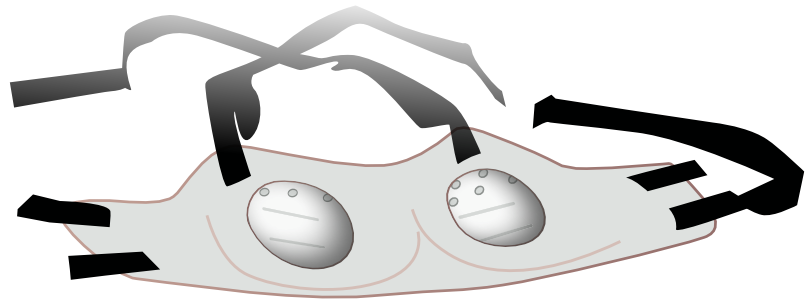

2

Remove back plate and pathologies support

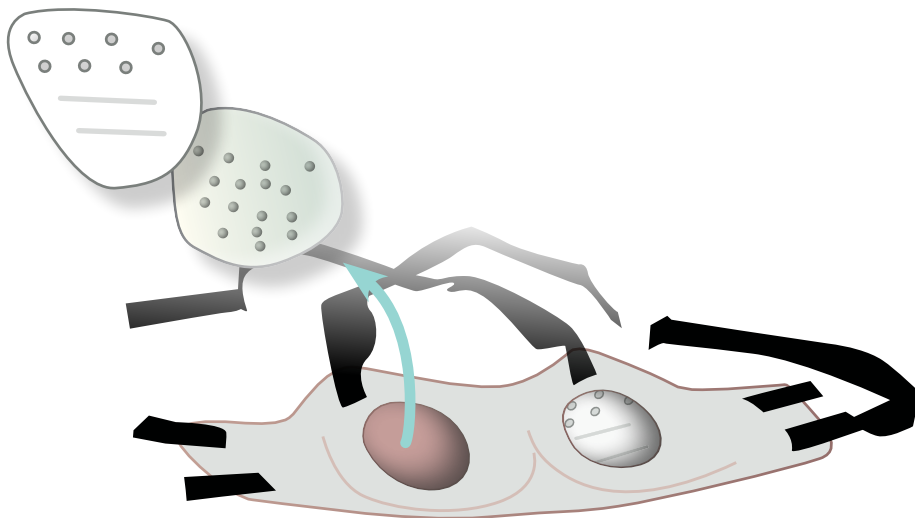


SET UP


SETTING UP PATHOLOGIES




Pathologies can be inserted in both back plate and pathologies support

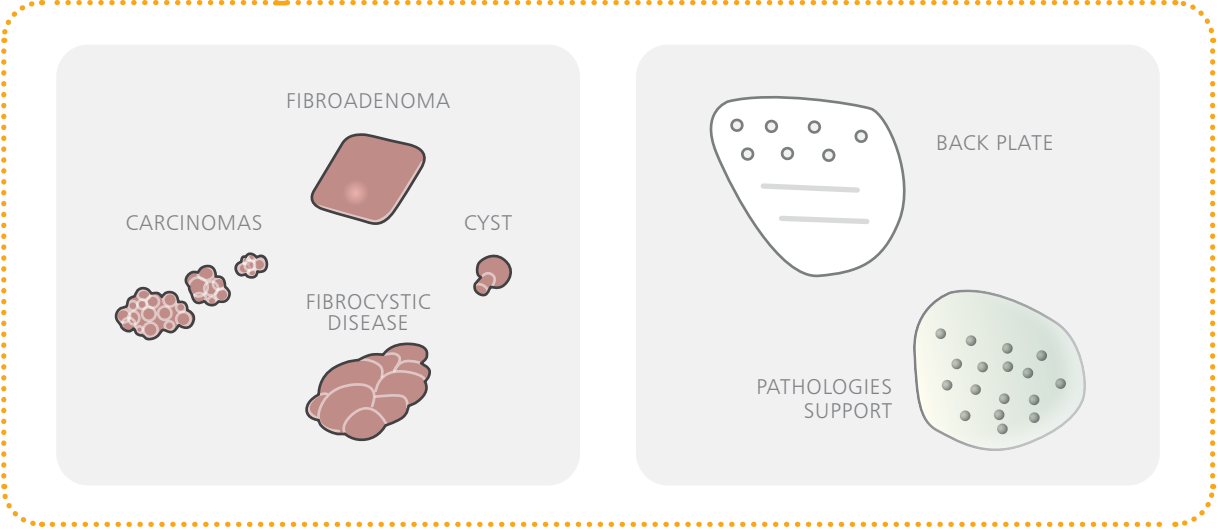


3

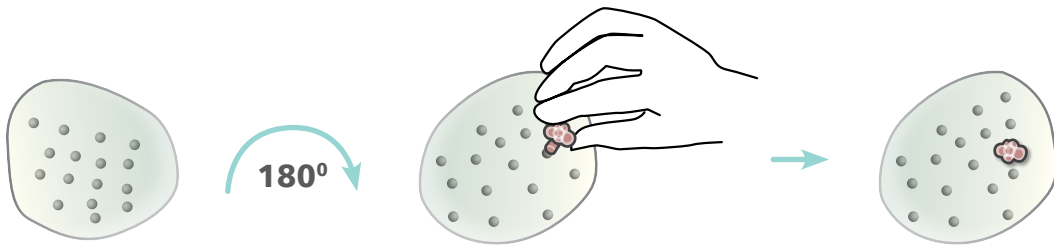

4

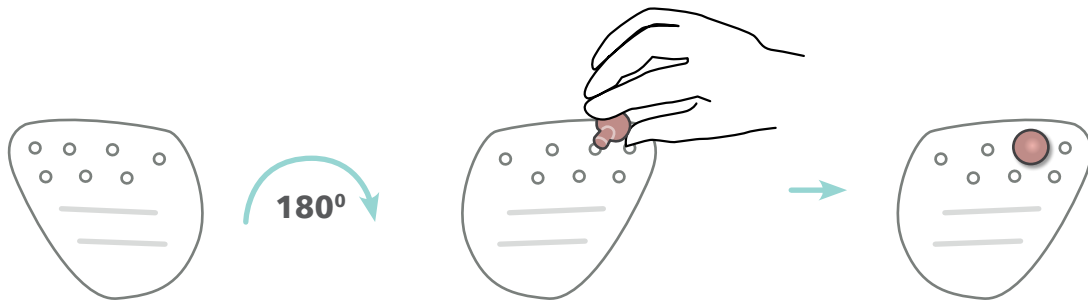


5

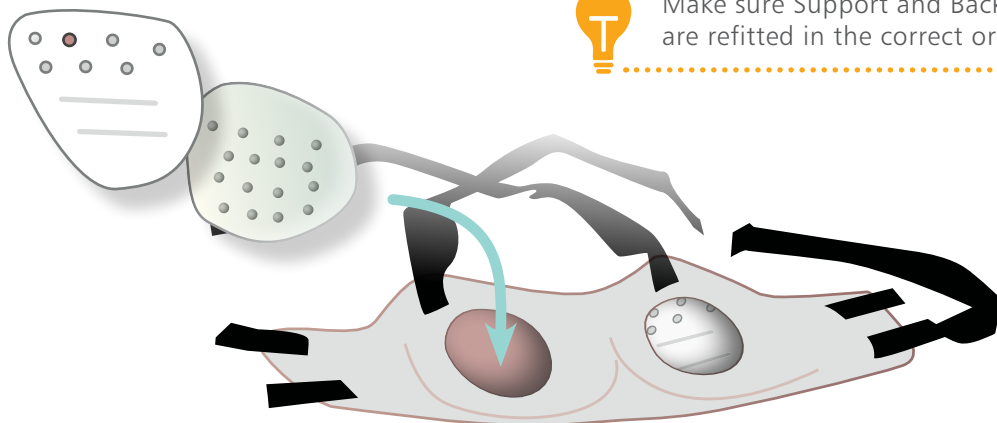


Make sure Support and Back Plate are refitted in the correct orientation

SET UP

REMOVAL & REFITTING BREAST INSERTS


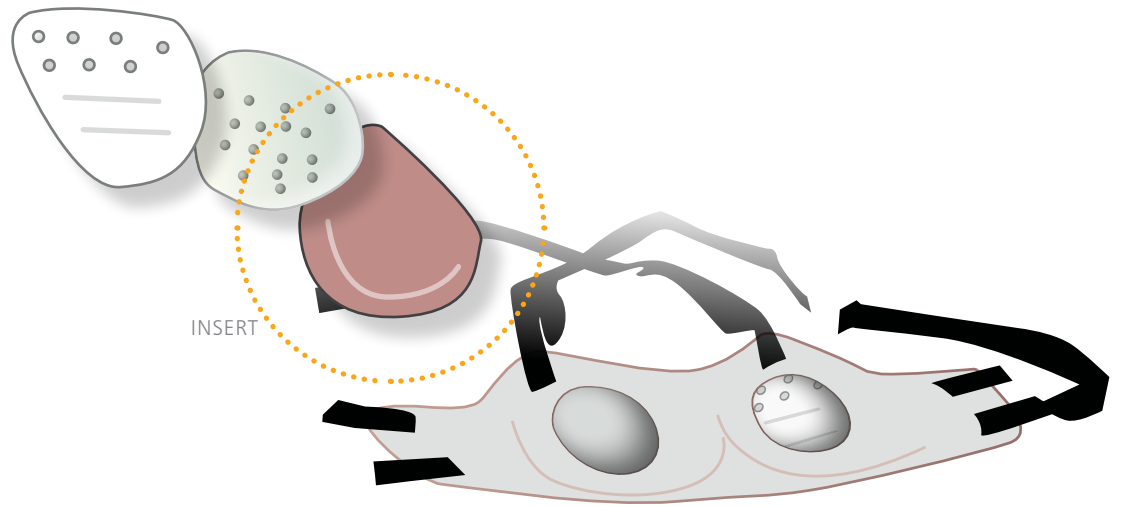


To ensure the breast appearance is smooth, please follow the steps below

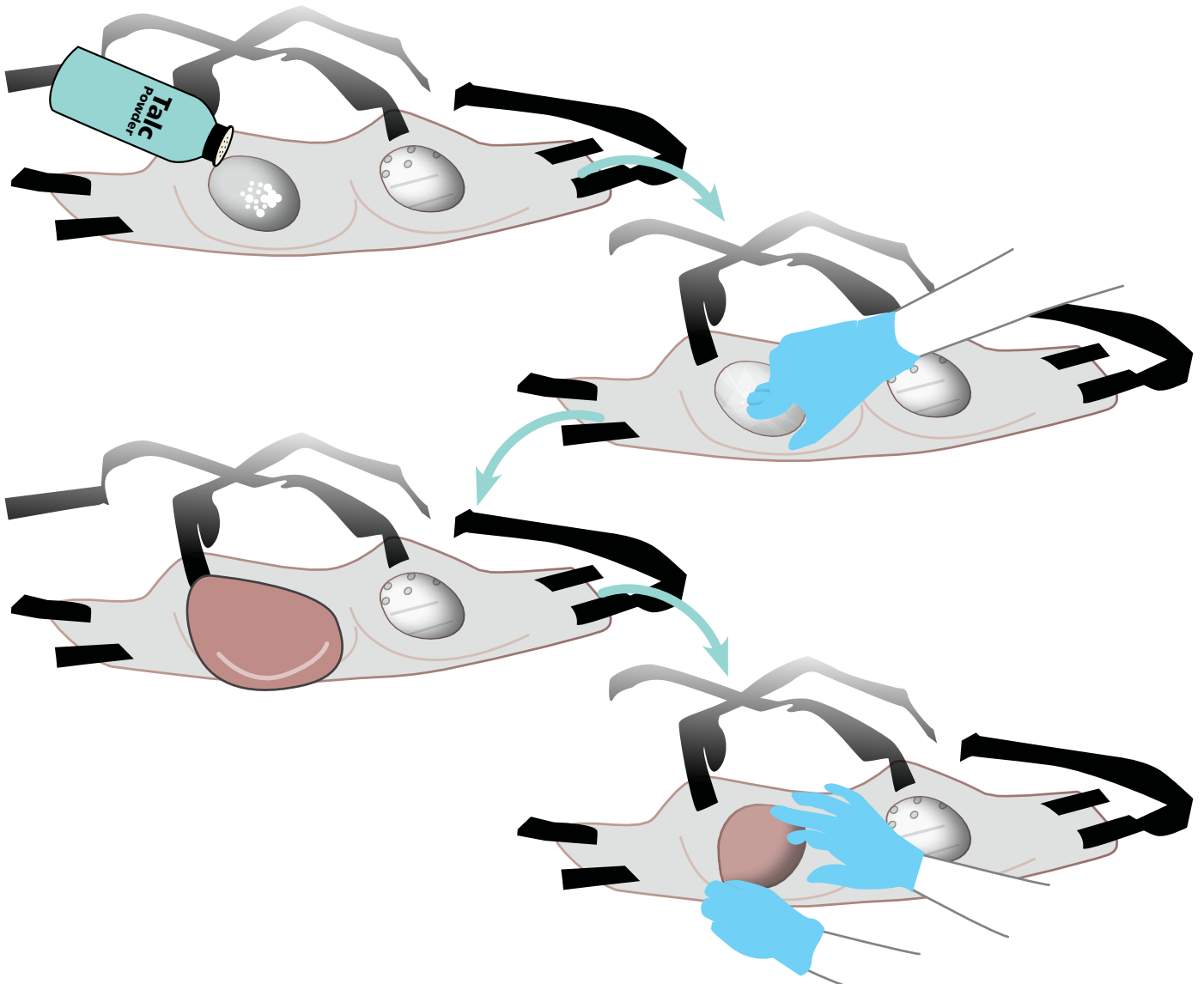

TRAINING

FITTING THE BREAST SKIN FOR WEARABLE USE

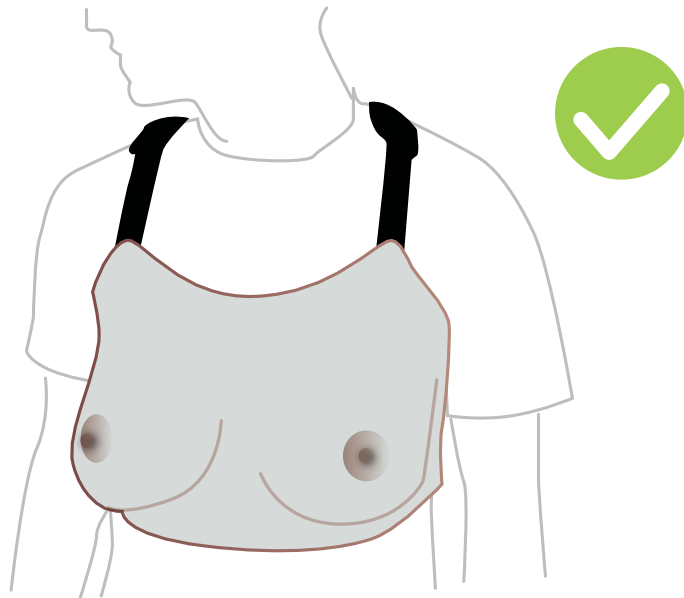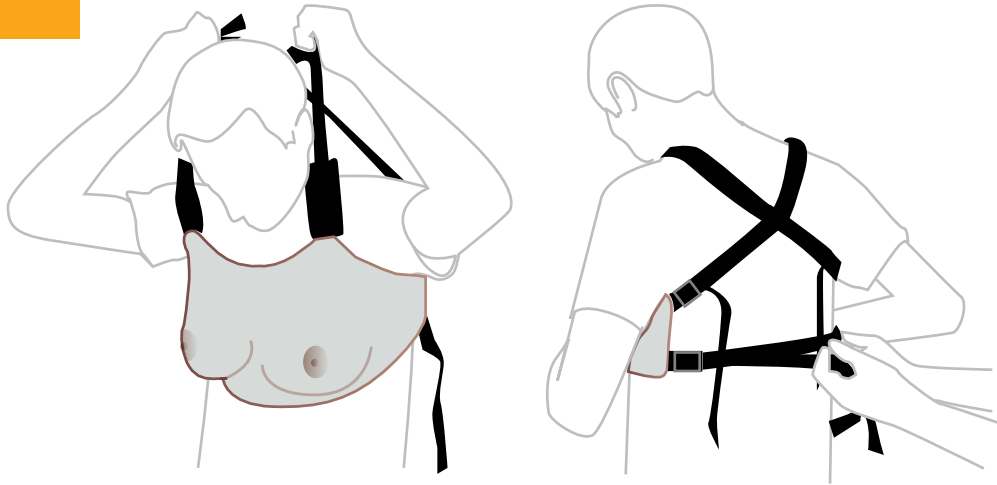

CLEANING
& STORAGE

Clean off any surface dirt with a damp, soft cloth, using only warm water and mild detergent



YOU MAY
ALSO BE
INTERESTED IN...

OPTIONAL TO PURCHASE:

Breast Examination Torso


40208



Lymph Node Pads

40203

41203

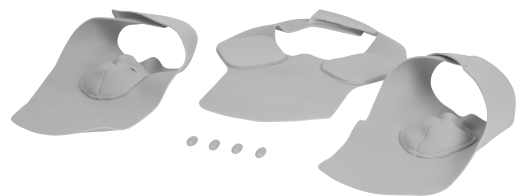

TAKE A LOOK
AT OUR **VIDEO**



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings



Part No: 365-973

Issue 1, March 2021
© 2021 Limbs & Things