




USER GUIDE

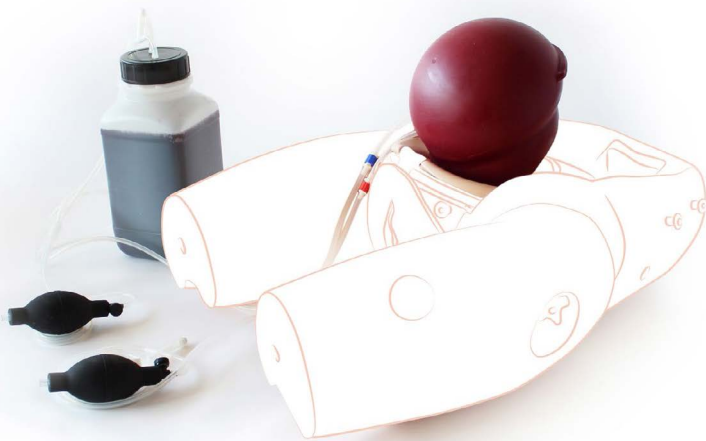

PPH MODULE PROMPT FLEX

This product is available
in light and dark skin tones

80101

80201

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	2
SAFETY & CLEANING INSTRUCTIONS	3
SET UP	4
• GETTING THE PROMPT FLEX READY	4
• FITTING THE PPH MODULE	6
• ATTACHING THE CONNECTORS & ABDOMEN	7
• PRESSURIZING BOGGY UTERUS	8
• ADDING BLOOD TO UTERUS	9
TRAINING	10
• USING THE MODULE	10
CLEANING & STORAGE	11

PACKAGE SUPPLIED

PPH UTERUS ASSEMBLY

80127

80227

CONCENTRATED VENOUS BLOOD - STARTER PACK

00020



PPH BLOOD CONTAINER

YOU'LL NEED (NOT SUPPLIED)

BIRTHING SIMULATOR PROMPT FLEX

80120

80220



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR PPH MODULE PROMPT FLEX

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Only use simulated blood supplied with this product. Use of non-recommended fluid products may weaken or damage the simulator.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.



STORAGE & TRANSPORTATION






Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

SET UP

GETTING THE PROMPT FLEX READY

Removing the abdominal skin

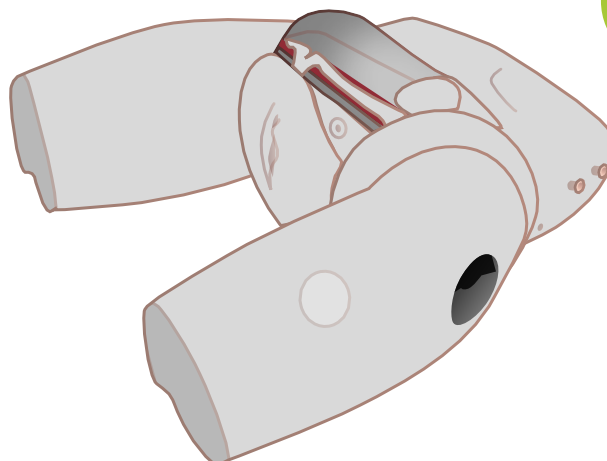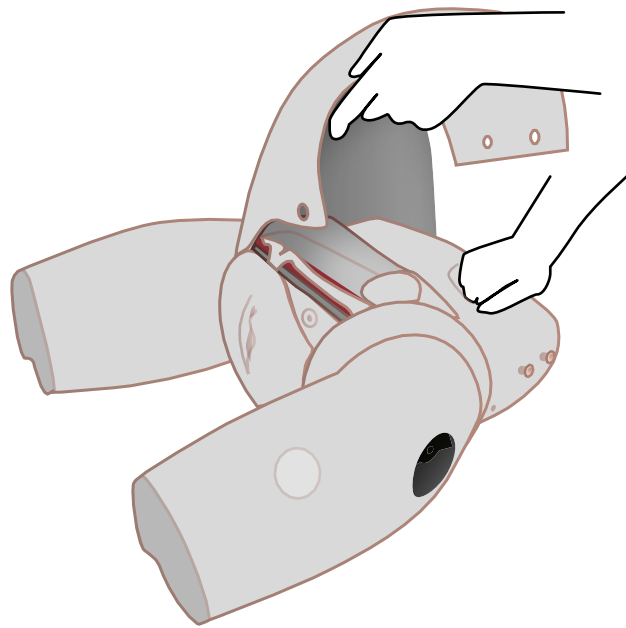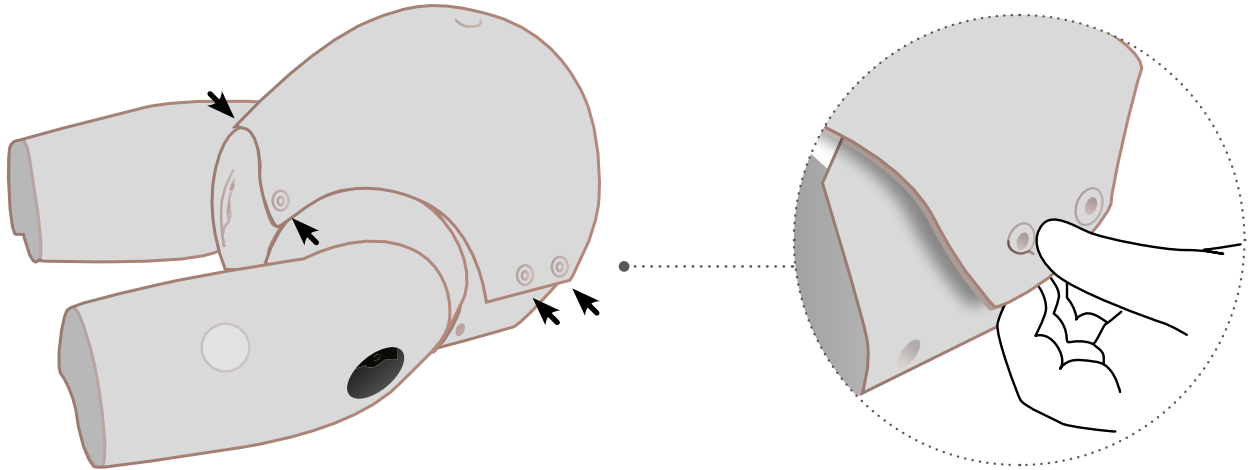


SET UP


GETTING THE PROMPT FLEX READY

Removing the ring clamp and cervix


1

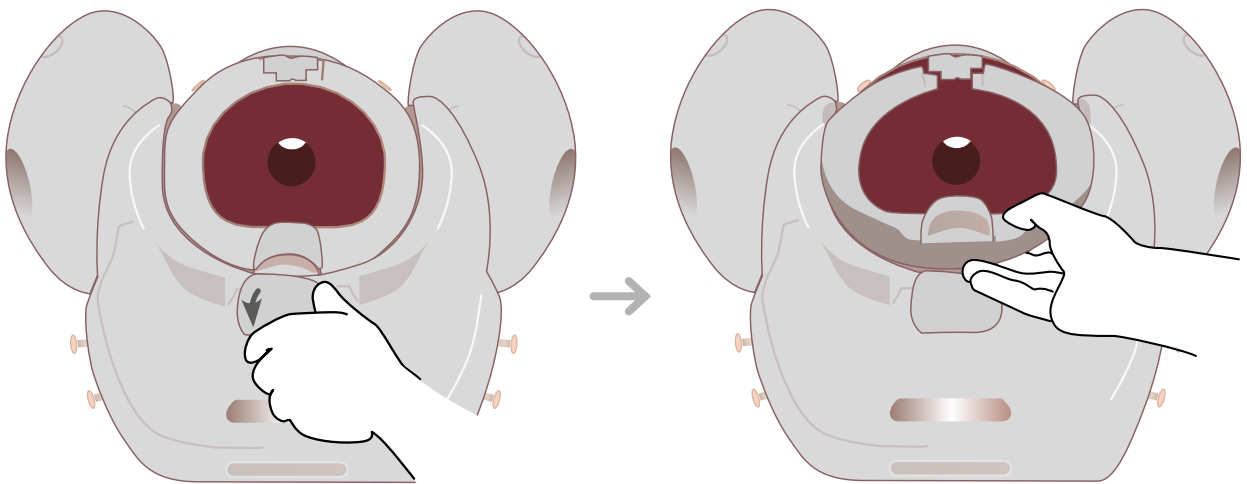


2

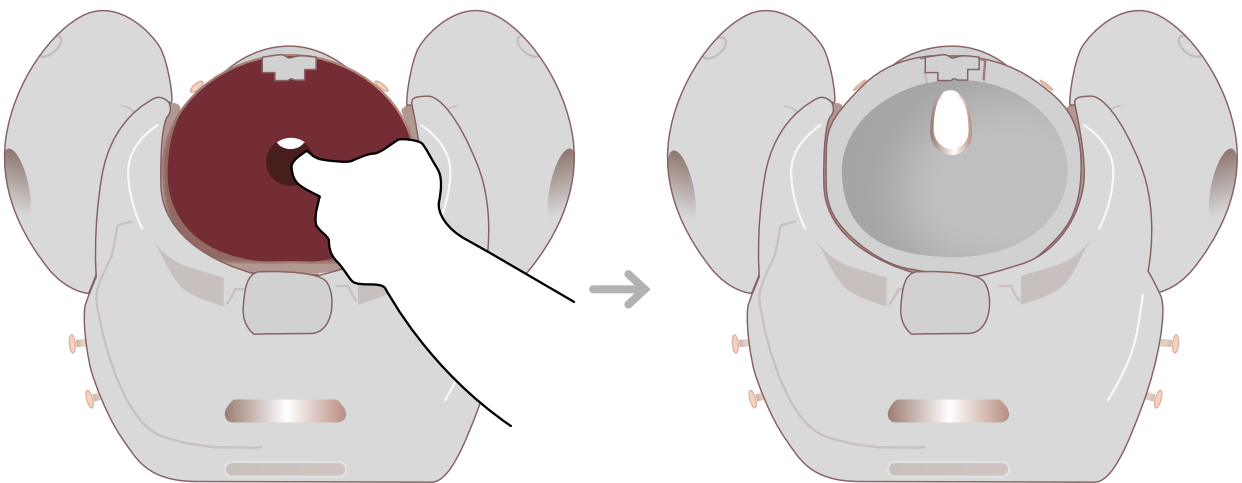

1



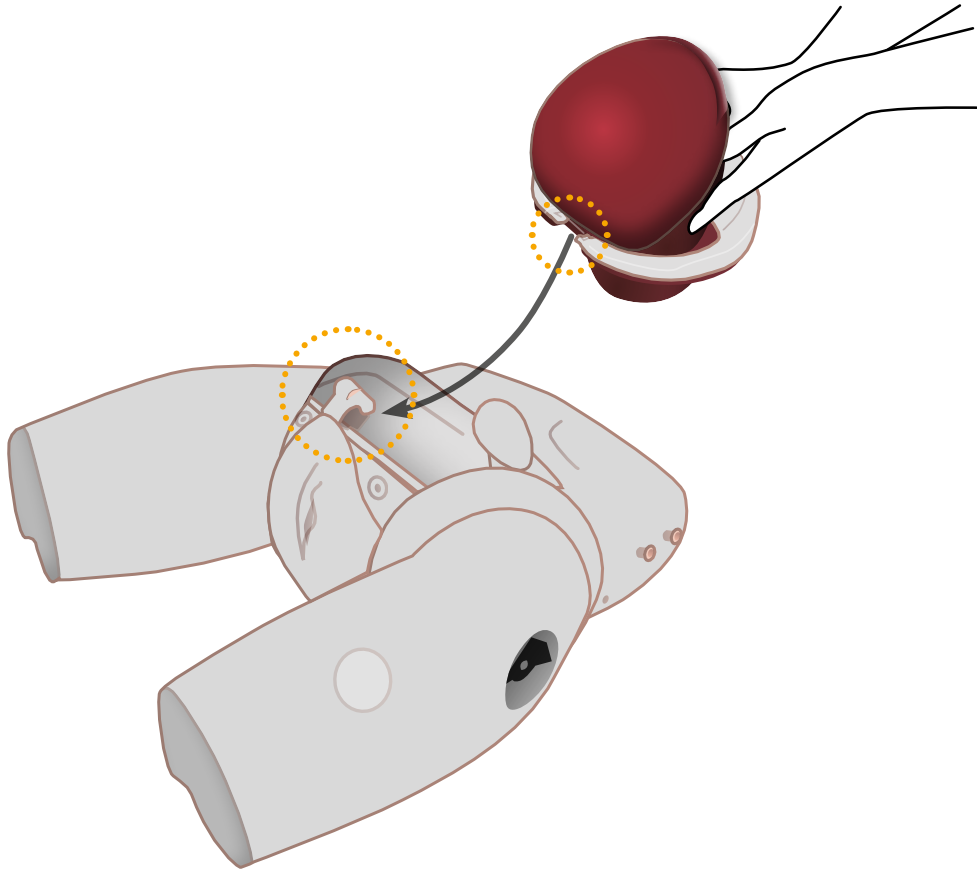


2

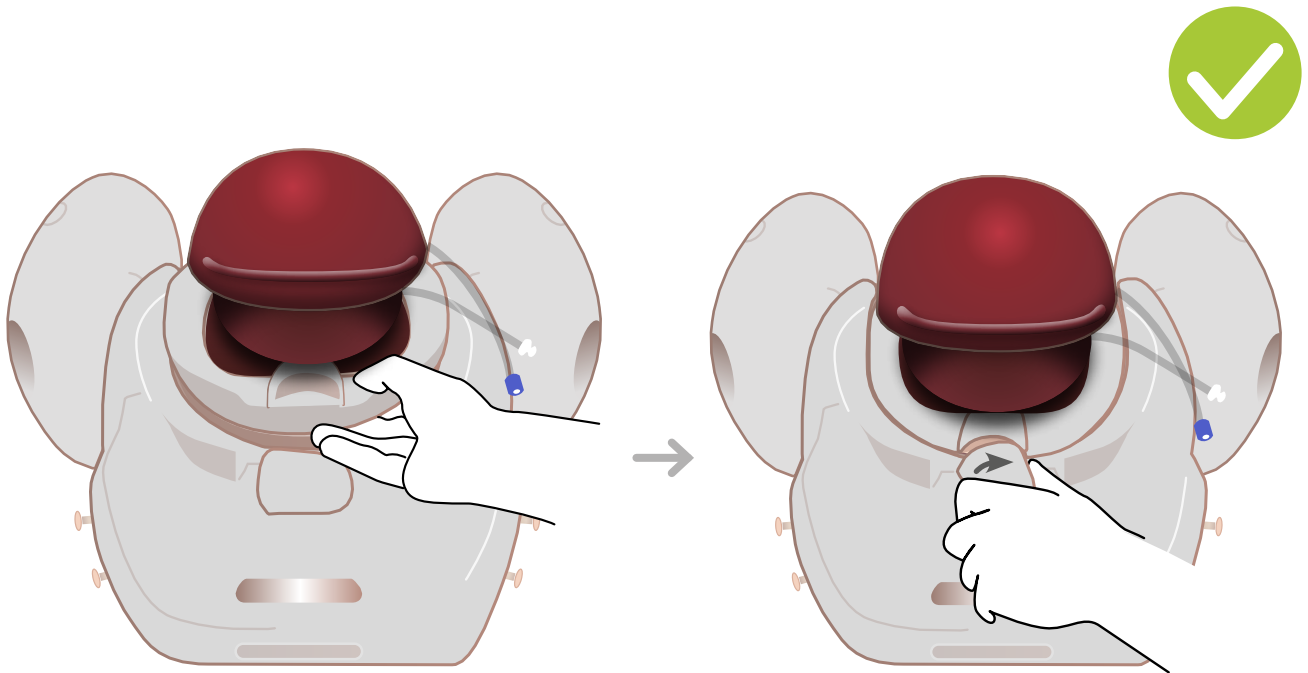


SET UP

ATTACHING THE CONNECTORS & ABDOMEN

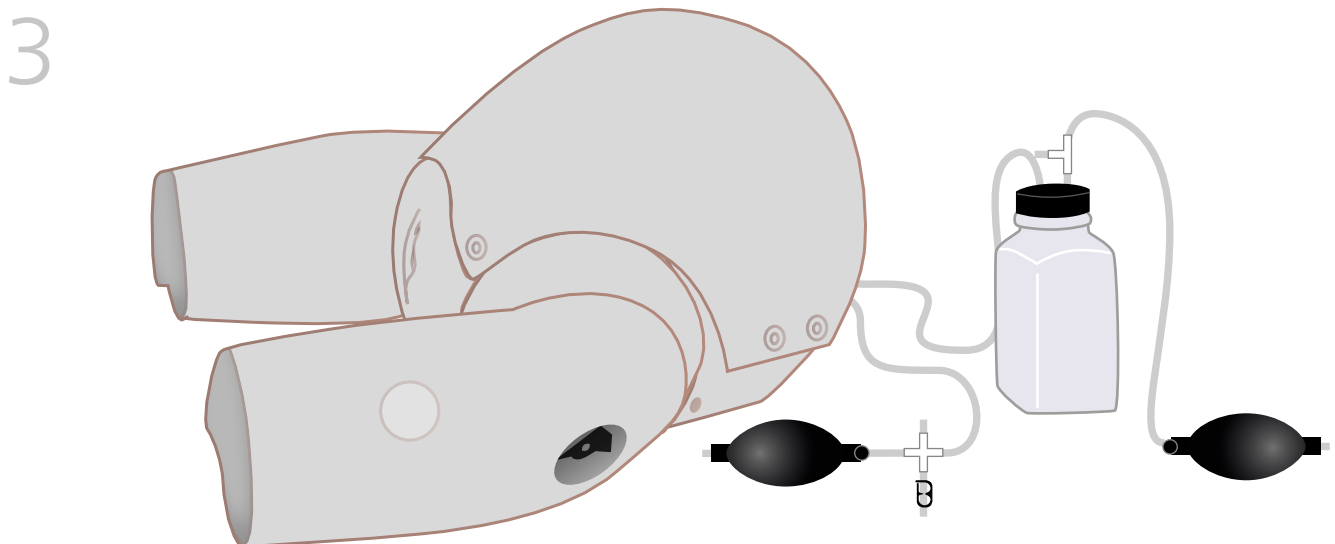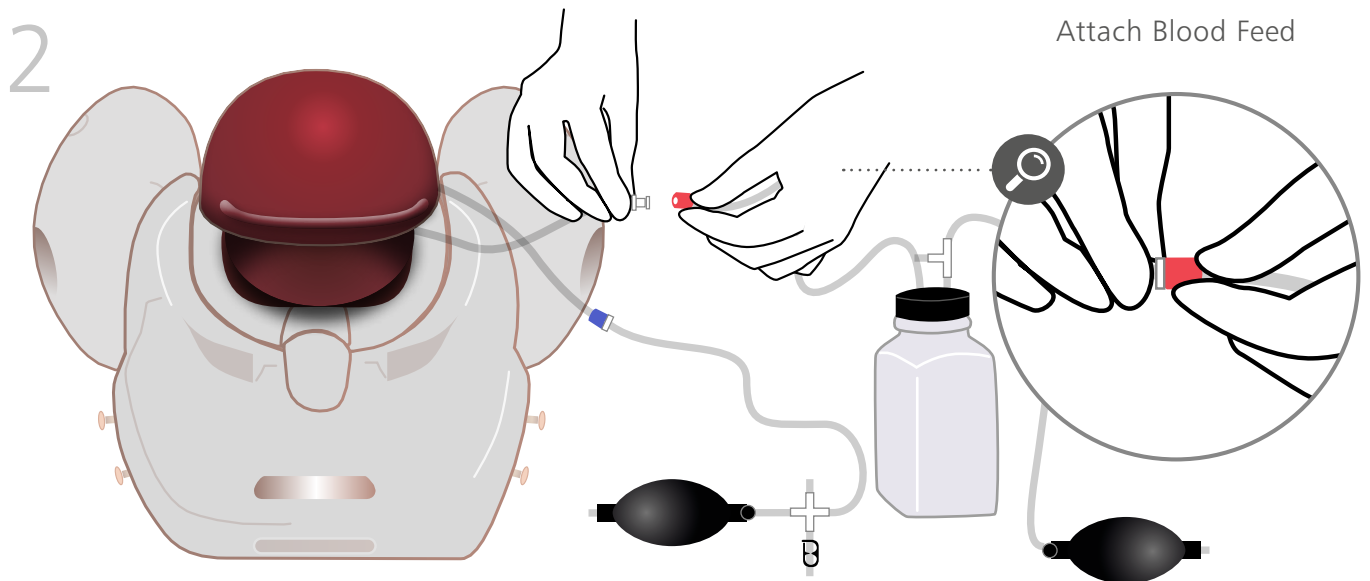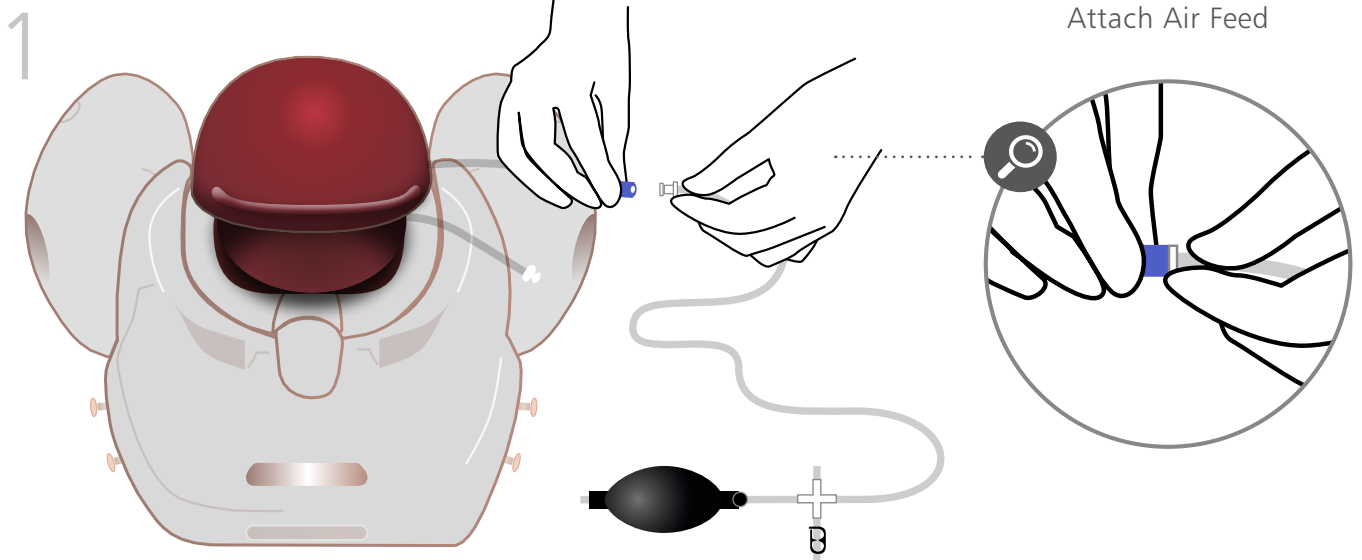



SET UP

PRESSURIZING BOGGY UTERUS

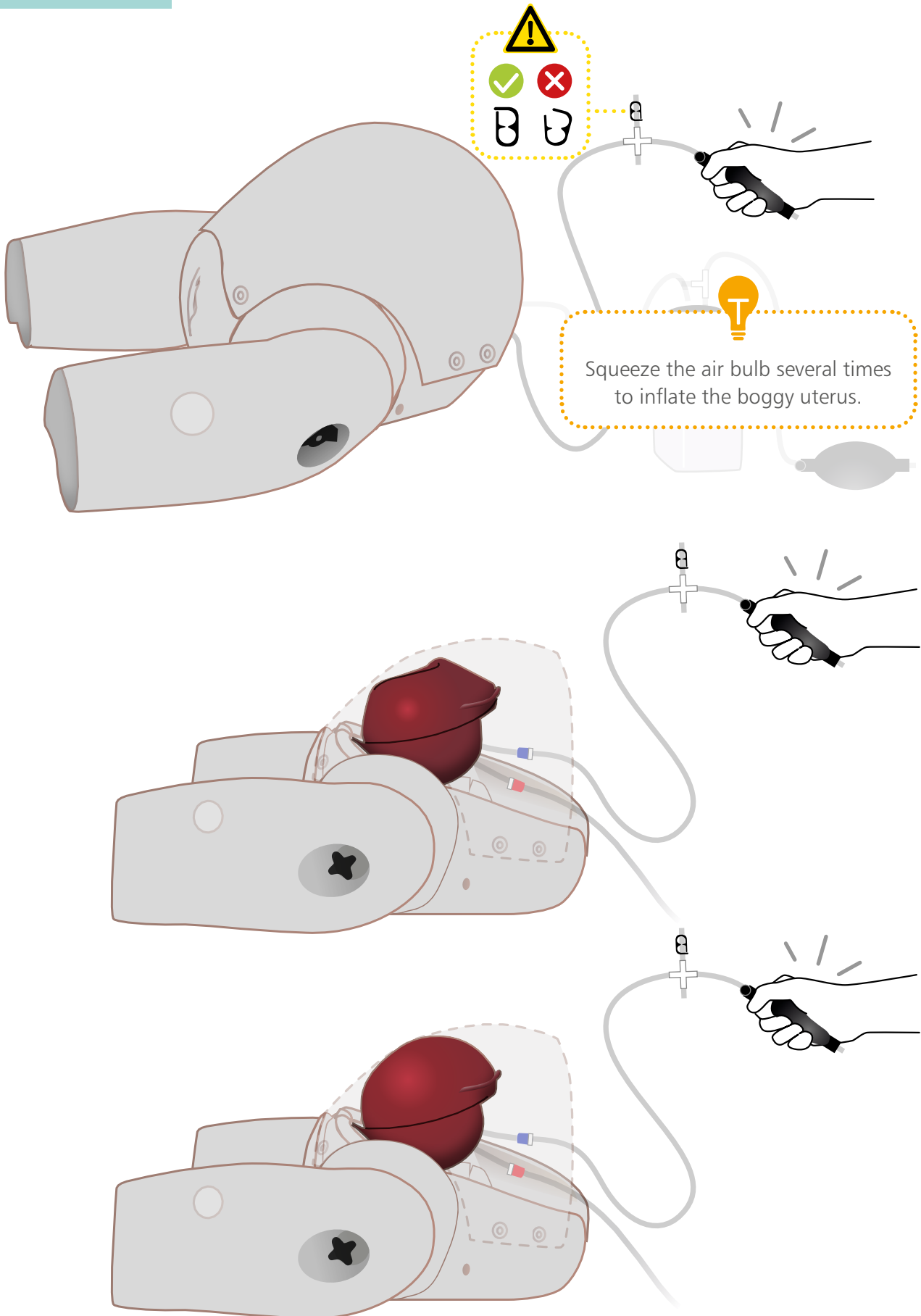

SET UP

ADDING BLOOD TO UTERUS

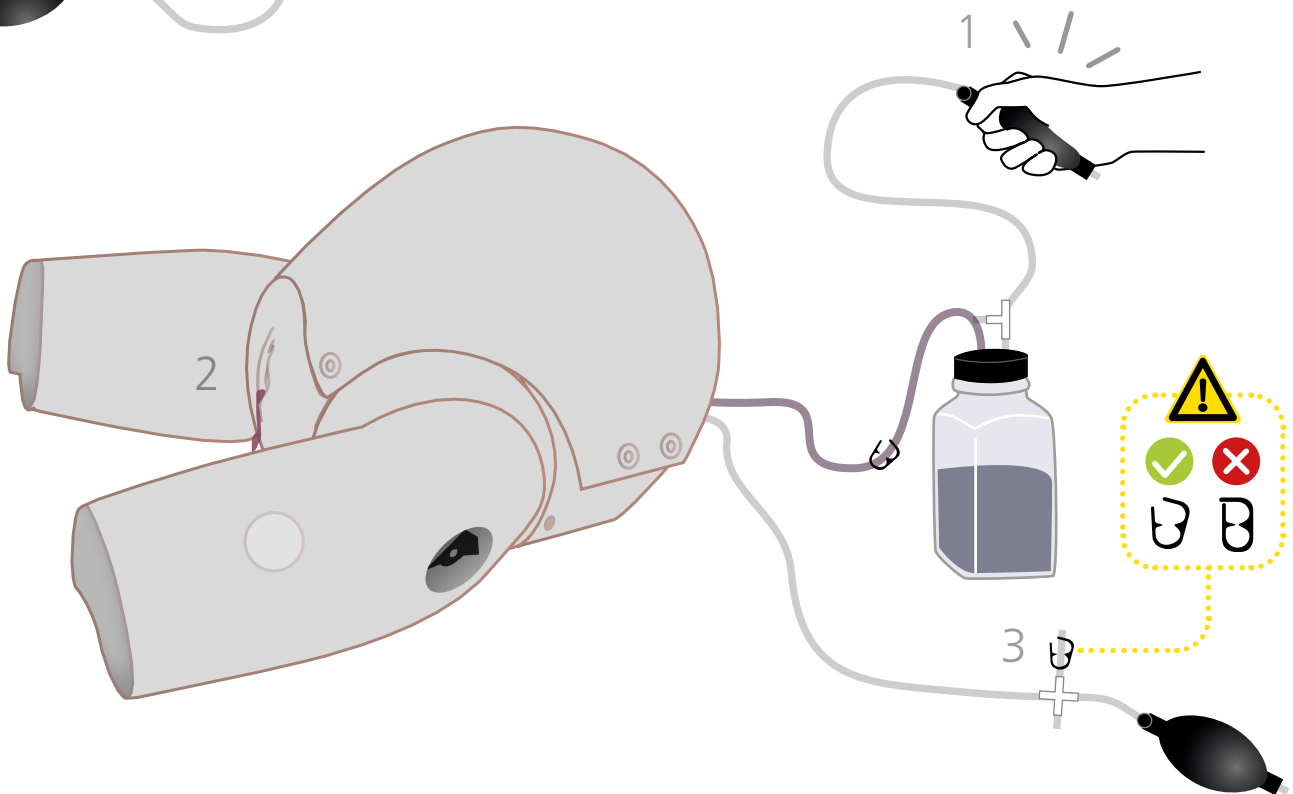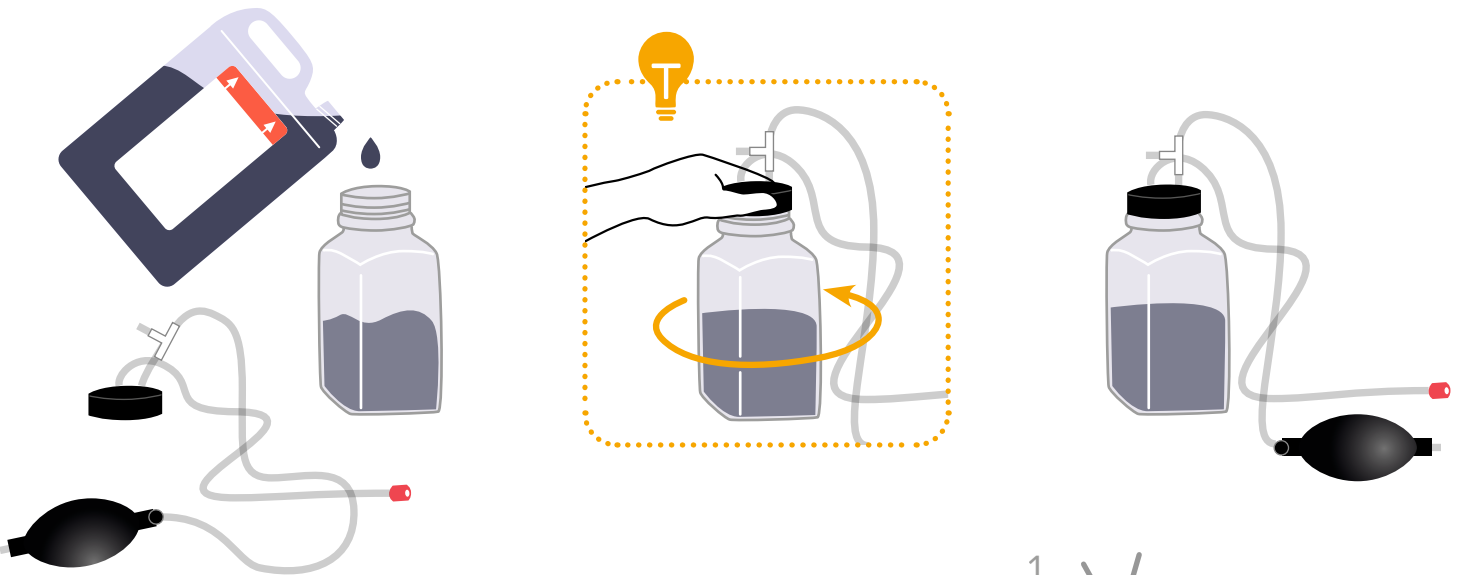


TRAINING


USING THE MODULE



The uterus can now be massaged.
Air will escape through the air release tube.

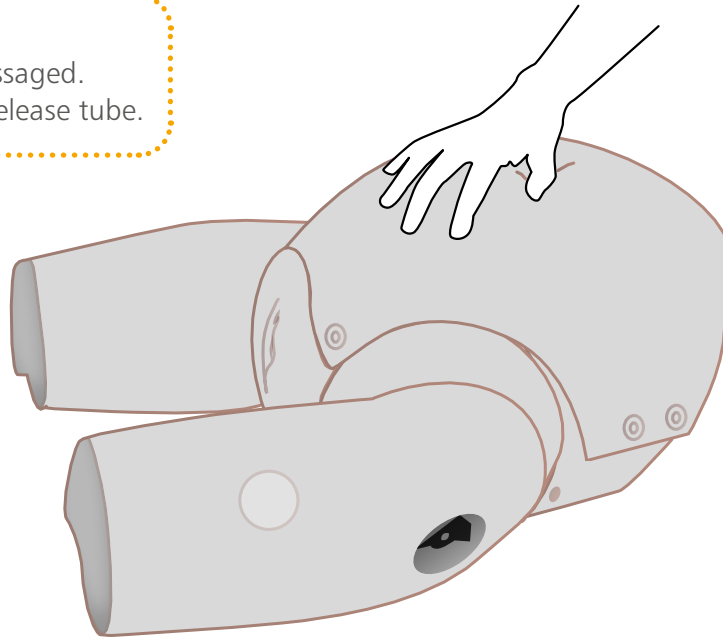




Once the uterus has been successfully massaged, close the clamp on the blood flow tube to stop the flow of blood.

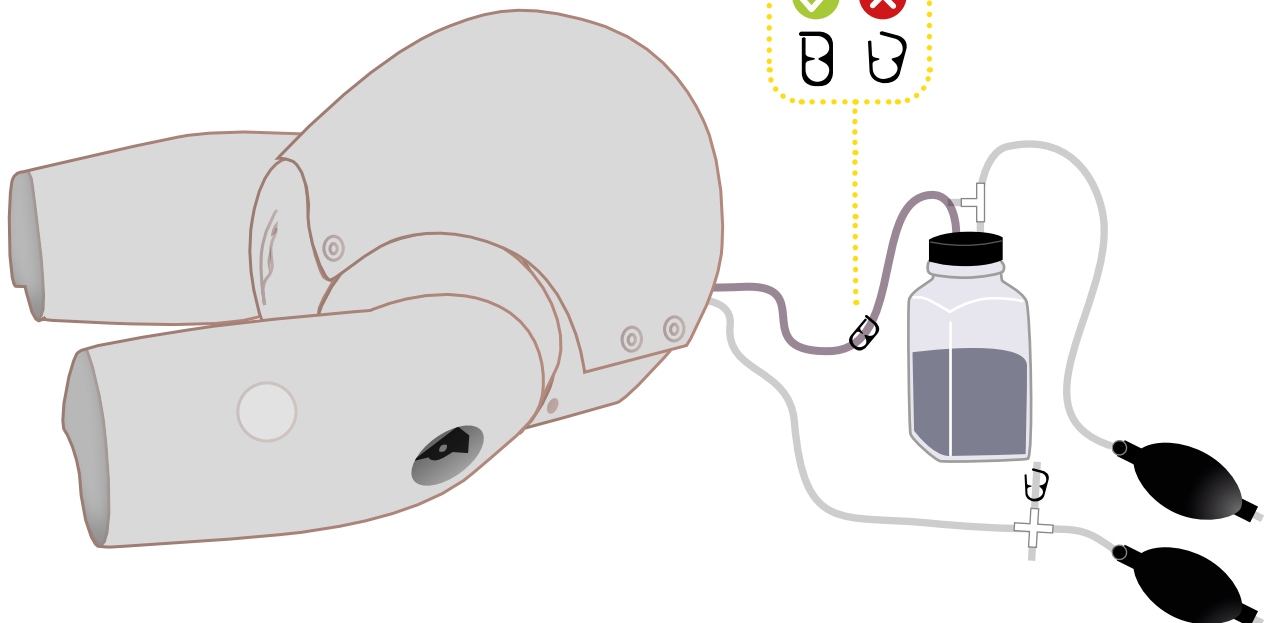



CLEANING & STORAGE



Use an empty syringe to remove any excess blood from the vagina after the drill.

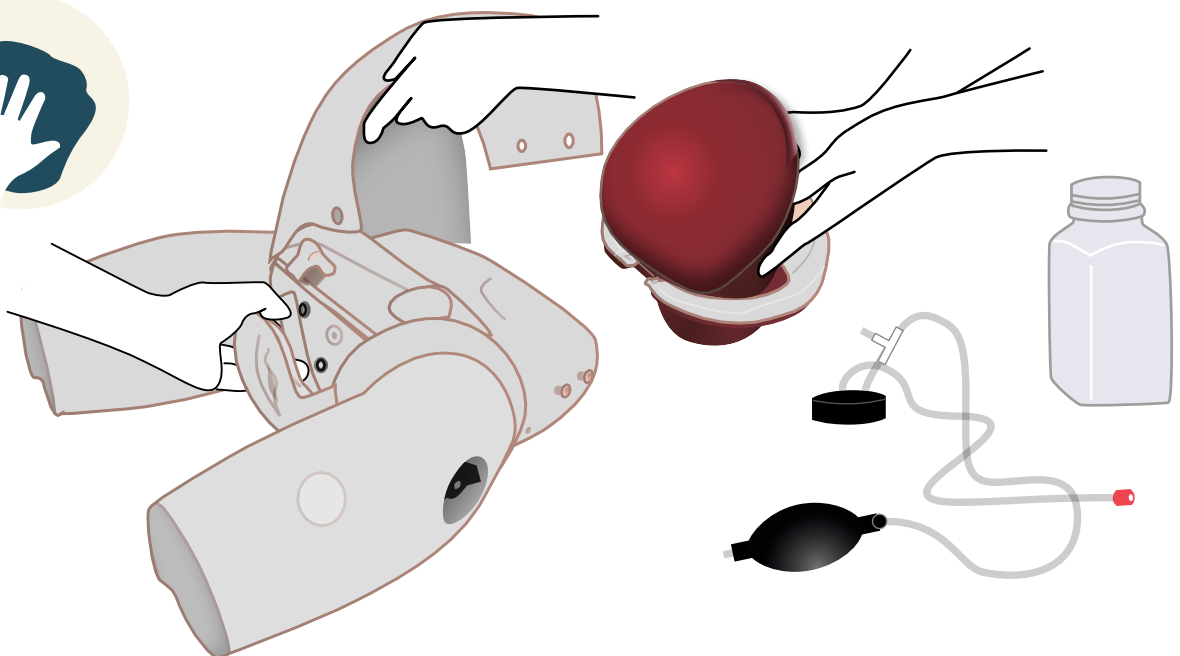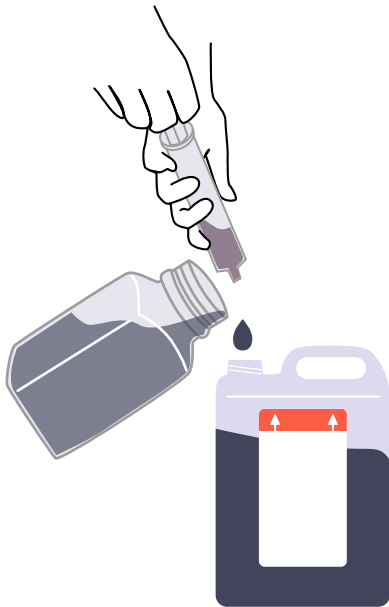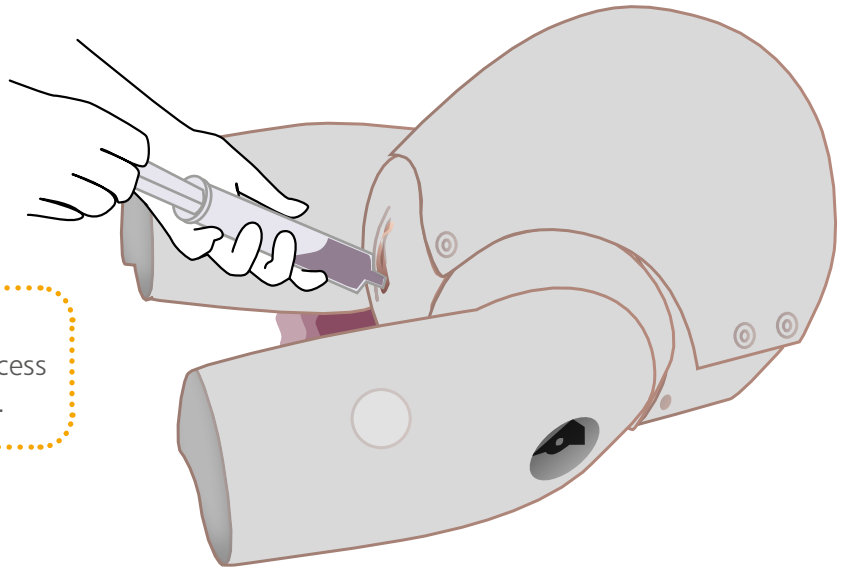

ADDITIONAL MODULES *
ALLOW YOU TO TAILOR
YOUR TRAINING NEEDS

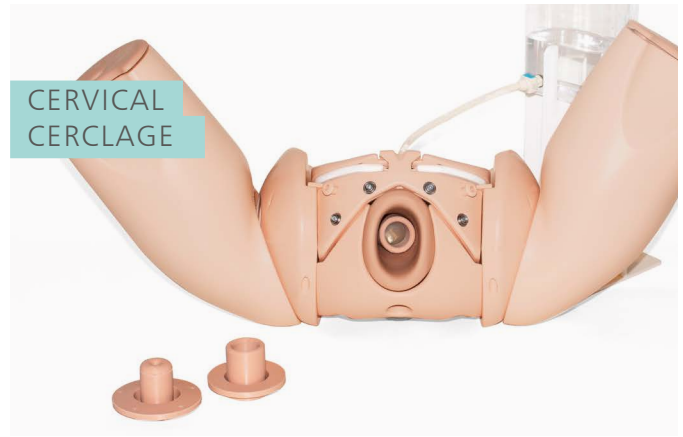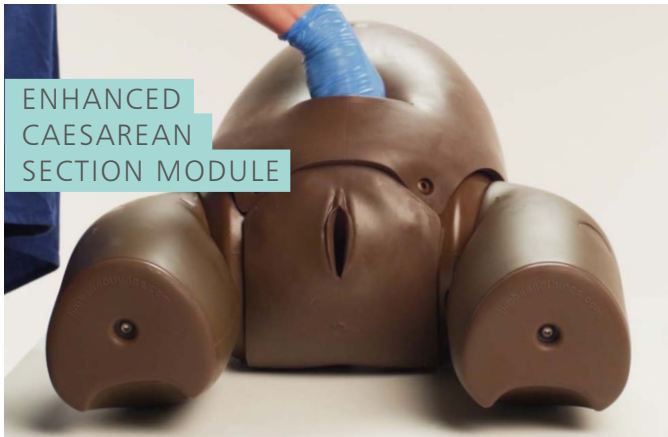

* All available in dark and light skin tones.